





APOLLO BATH

ELEVATING SENIOR HEALTH...WITH STYLE

Advantage™ Bathing System Parts Manual

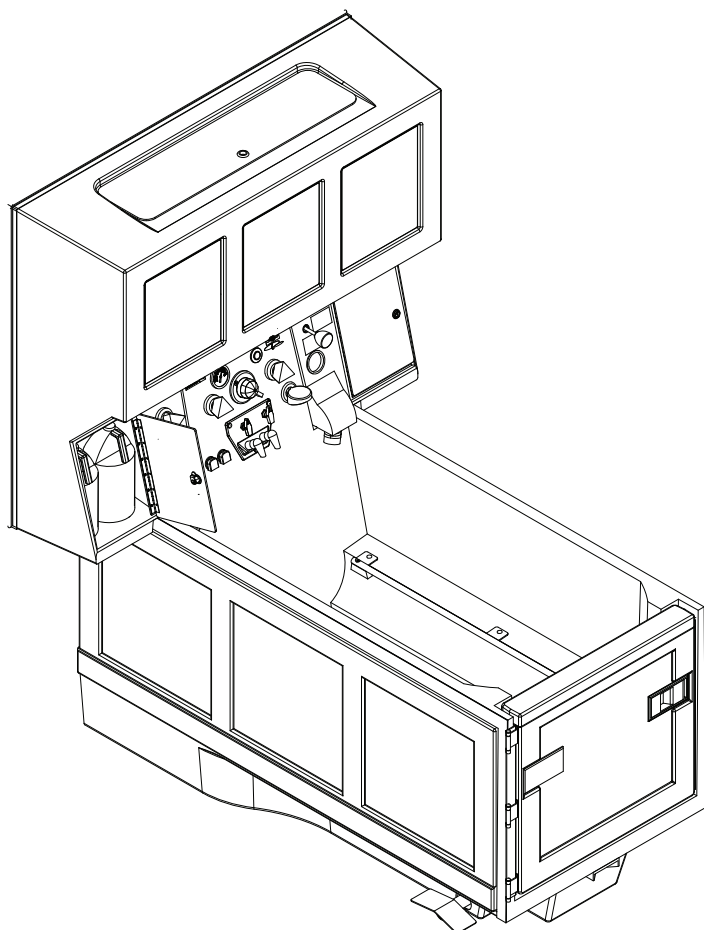

6000 Series

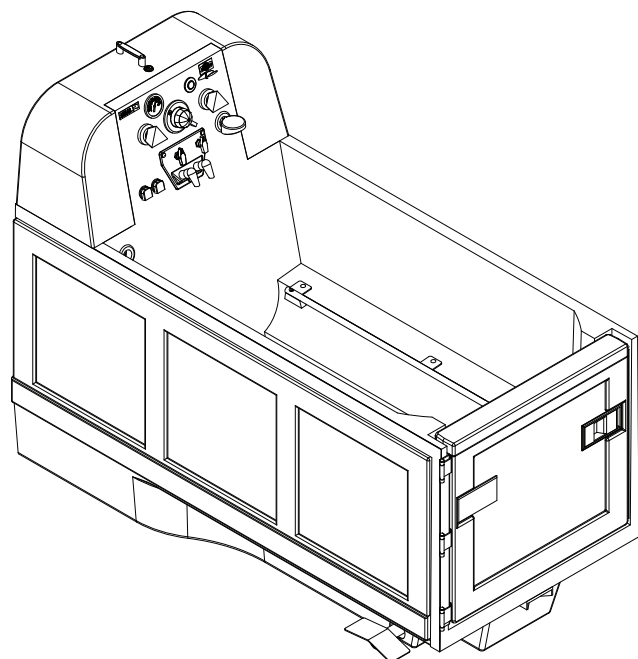

6300 Series

450 Main Street, Somerset, WI 54025

Phone: 715.247.5625 Fax: 715.247.3424 <https://apollobath.com>

The information in this manual is subject to change without notice.

In no event will Apollo Corporation be liable for technical or editorial omissions made herein; nor for direct, special, incidental, or consequential damages resulting from the furnishing, performance, or use of this material.

This manual is copyrighted with all rights reserved. Under the copyright laws, this manual may not be copied, in whole or part, without the written consent of Apollo Corporation.

Product Identification

Product: _____

Model: _____

Serial Number

Tub: _____

Cabinet/Shroud: _____

Rapid Fill™ Reservoir: _____

Level Glide™ Transfer System: _____

Mobile Digital Scale: _____

Optional Features

☐ Remedy® UV system

For Assistance Contact:

Apollo Corporation

450 Main St

Somerset, WI 54025

+1 (715) 247-5625


Fax +1 (715) 247-3424

techservice@apollobath.com


<https://apollobath.com>

Table of Contents

Advantage™ Whirlpool Bathing System.....	5
Whirlpool Control Panel (2009 - 2014).....	6
Whirlpool Control Panel (2014 - Present)	7
Whirlpool Control Panel Valves (2009 - 2014).....	8
Whirlpool Control Panel Valves (2014 - Present)	9
Whirlpool Cleaning/Disinfecting System	10
Whirlpool Pump and Suction Assemblies	11
Whirlpool Jet Assemblies	12
Remedy® UV system.....	13
Advantage™ Air Spa Bathing System.....	14
Air Spa Control Panel (2009 - 2014)	15
Air Spa Control Panel (2014 - Present)	16
Air Spa Control Panel Valves (2009 - 2014).....	17
Air Spa Control Panel Valves (2014 - Present)	18
Air Spa Cleaning/Disinfecting System	19
Air Spa Blower & Manifold	20
Air Spa Full Size / Space Saver System.....	21
Advantage™ Series Bathing System Assemblies	22
Tub Assembly.....	23
Tub Door.....	24
Tub Drain Assembly.....	25
Slide Valve Drain Assembly - 200344	26
Tub Shroud Assembly - 200348	27
Reservoir Cabinet Shroud	28
Reservoir Tank Assembly - 210050	29
Level Glide™ Transfer System.....	30
Deluxe Chair Assembly - 220044	31
Deluxe Chair Assembly - 220044	32
Carrier Assembly - 220079	33
Carrier Assembly w/Scale - 220080	34
Carrier Assembly - 220041	35
Carrier Assembly w/Scale - 220042	36
Lock-In Chair Assembly – Full Size (Tub view).....	37
Lock-In Chair Assembly – Full Size - 220074.....	38
Lock-In Chair Assembly – Space Saver (Tub view)	39
Lock-In Chair Assembly – Space Saver - 220047	40
Scale Assembly (2005 - 2014).....	41
Scale Assembly - FFI (2014 - Present)	42

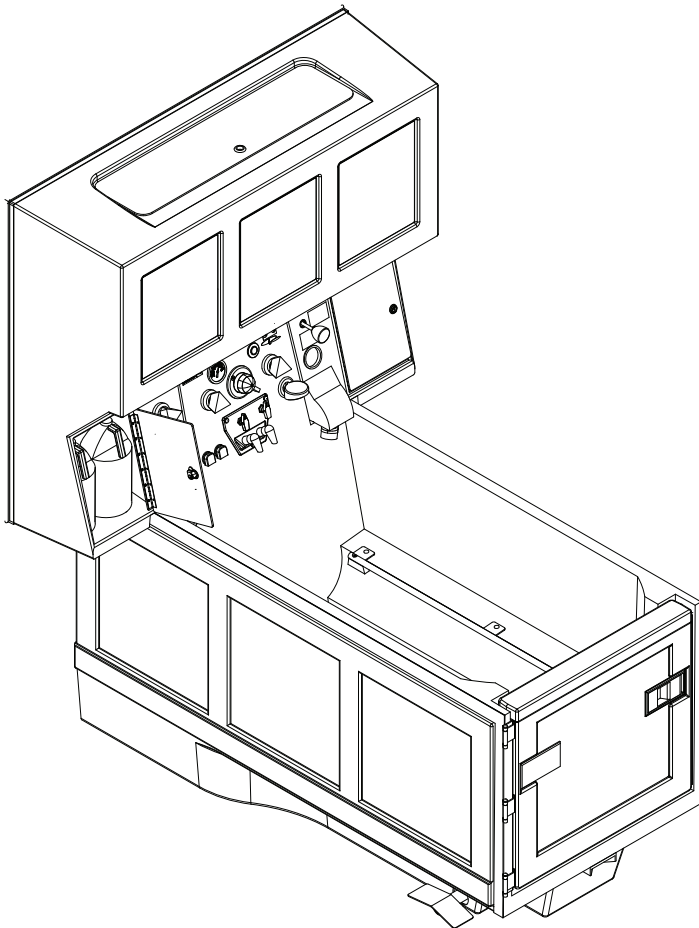


WP6000 Series

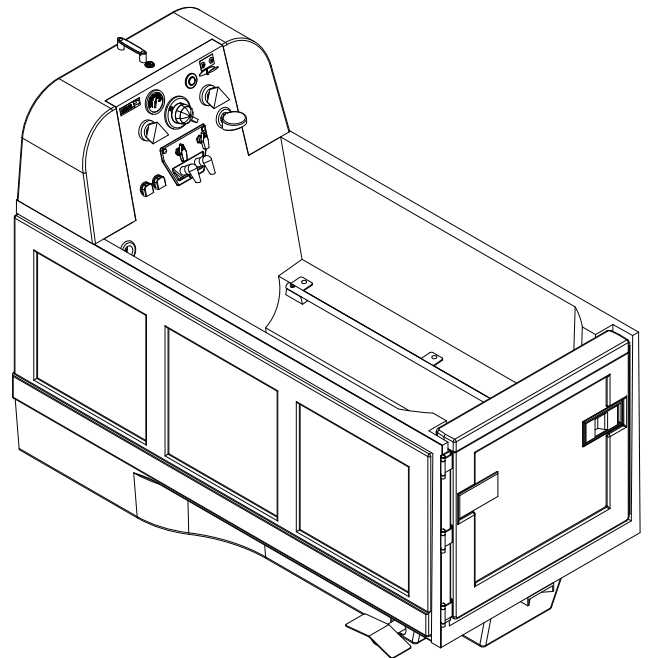

WP6300 Series


Whirlpool Control Panel (2009 - 2014)

Item	Part #	Description	Item	Part #	Description
1	320329	Chrome Knob	11	320006	Shower Wand
2	510087	Valve Cartridge 3/4" 90 degree	12	510082	Valve Cartridge 1/2" 90 Degree
3	510088	Renewable Seat Set 3/4"	13	510084	Valve Seat 1/2"
4	320295	3-Way Valve	14	320420	Fill Spouts
4	200317	3-Way Valve with 90 degree elbows	15	320296	2-Way Valve
5	510066	Rada 222 Control Knob	15	200313	2-Way Valve with 90 degree elbows
6	510071	Rada 222 Radatherm Cartridge	16	200440	Shampoo/Body Wash Pump
7	350336	Thermometer	17	200441	Bath Oil Pump
8	500132	Air Button	18	400856	Solutions Panel
9	200229	Remedy® UV system Indicator	Kit	500147	Skin Care Pump Kit (items 16 & 17)
10	320005	Shower Hose			

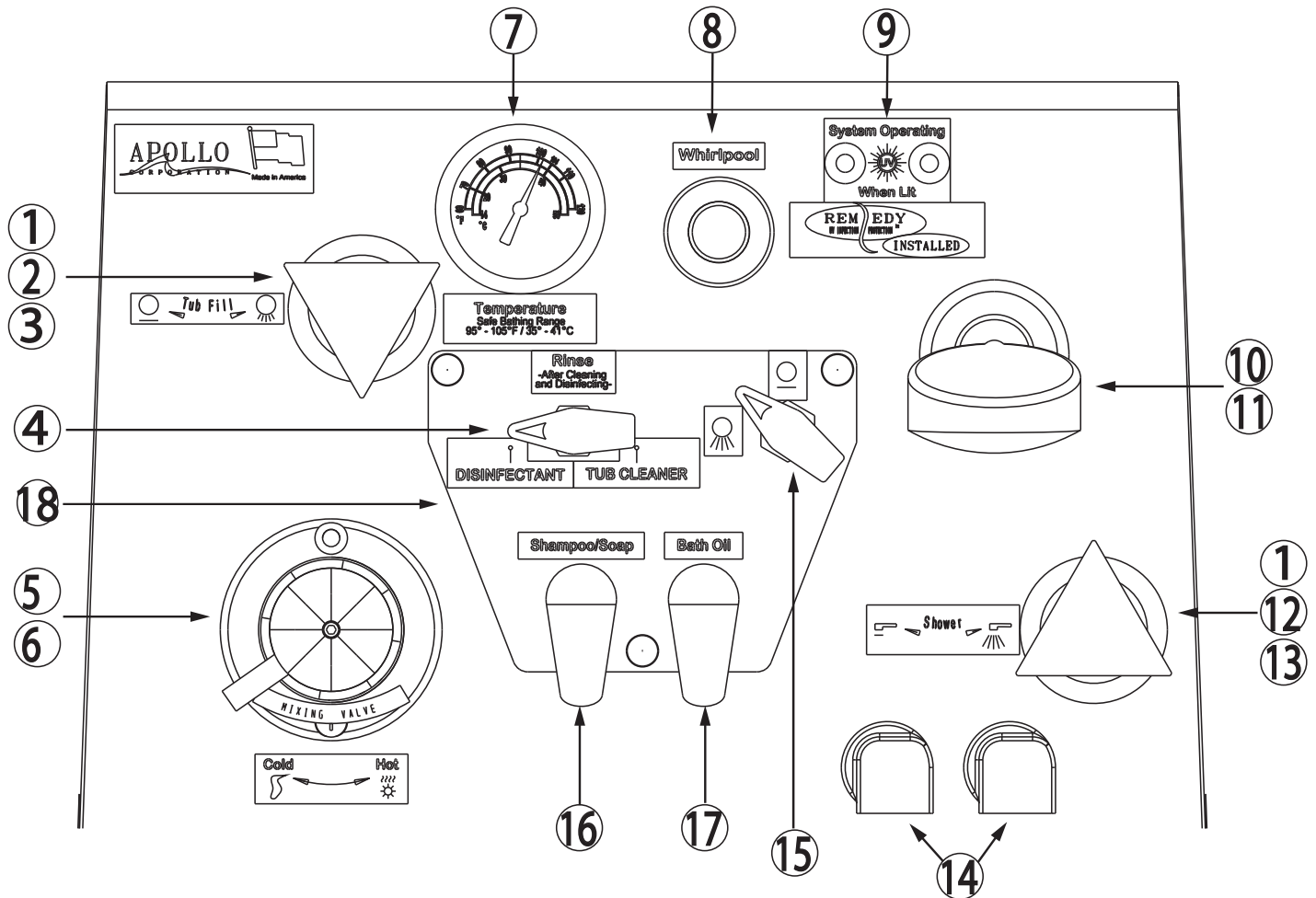

Whirlpool Control Panel (2014 - Present)


Item	Part #	Description	Item	Part #	Description
1	320329	Chrome Knob	11	320006	Shower Wand
2	510087	Valve Cartridge 3/4" 90 degree	12	320420	Fill Spouts
3	510088	Renewable Seat Set 3/4"	13	320296	2-Way Valve
4	320522	3-Way Valve	13	200313	2-Way Valve with 90 degree elbows
4	200435	3-Way Valve with 90 degree elbows	14	200440	Shampoo/Body Wash Pump
5	510066	Rada 222 Control Knob	15	200441	Bath Oil Pump
6	510071	Rada 222 Radatherm Cartridge	16	400856	Solutions Panel
7	350336	Thermometer	Kit	500147	Skin Care Pump Kit (items 14 & 15)
8	500132	Air Button			
9	200229	Remedy® UV system Indicator			
10	320005	Shower Hose			

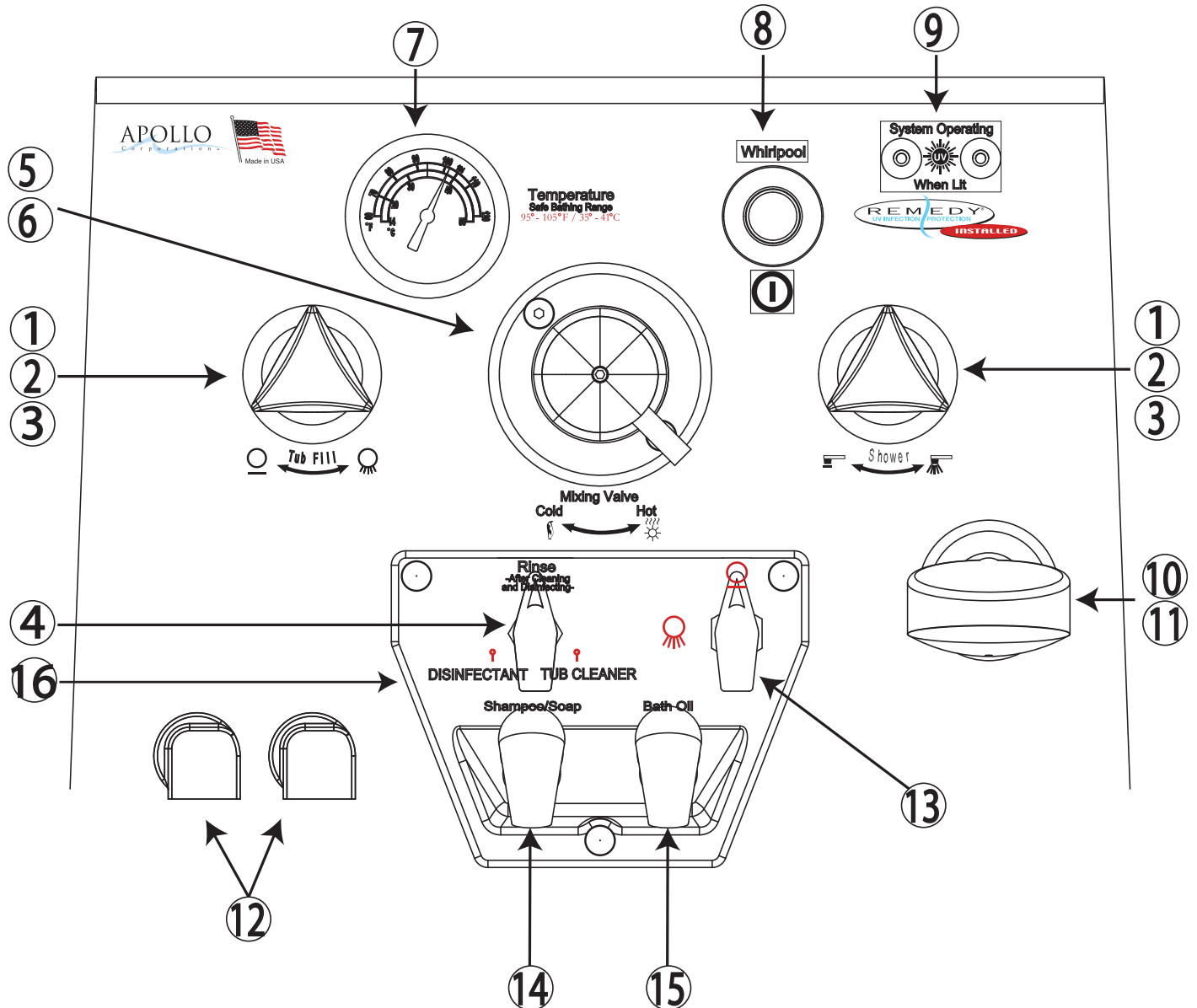

Whirlpool Control Panel Valves (2009 - 2014)


Item	Part #	Description
1	320329	Chrome Knob
2	510090	Handle Connection Set
3	510091	Extension Sleeve
4	320423	Classic Flange
5	510086	Extension Spindle
6	320420	Fill Spouts
7	510067	Handle Locking Screw Rada 222
8	510066	Rada 222 Control Knob

Item	Part #	Description
9	450073	Rada Deck Plate 222
10	350336	Thermometer
11	510071	Rada 222 Radatherm Cartridge
12	510082	Valve Cartridge 1/2" 90 Degree
13	510084	Valve Seat 1/2"
14	510087	Valve Cartridge 3/4" 90 degree
15	510088	Renewable Seat Set 3/4"
16	320003C	Vacuum Breaker (ASSE 1014)

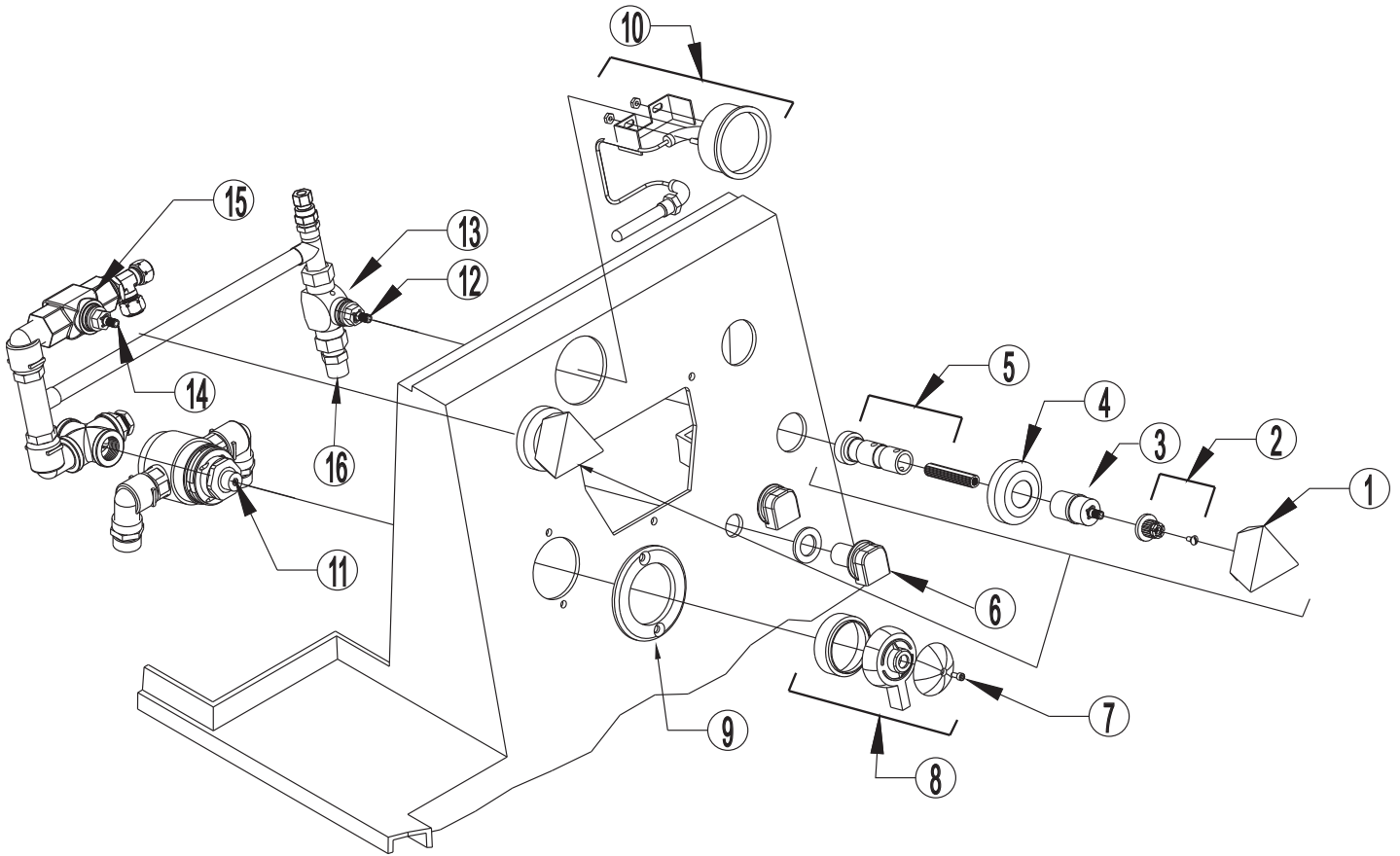


Whirlpool Control Panel Valves (2014 - Present)

Item	Part #	Description	Item	Part #	Description
1	320329	Chrome Knob	9	450073	Rada Deck Plate 222
2	510090	Handle Connection Set	10	350336	Thermometer
3	510091	Extension Sleeve	11	510071	Rada 222 Radatherm Cartridge
4	320423	Classic Flange	12	510087	Valve Cartridge 3/4" 90 degree
5	510086	Extension Spindle	13	510088	Renewable Seat Set 3/4"
6	320420	Fill Spouts	14	320003C	Vacuum Breaker (ASSE 1014)
7	510067	Handle Locking Screw Rada 222	15	500352	Water Manifold Assembly
8	510066	Rada 222 Control Knob			

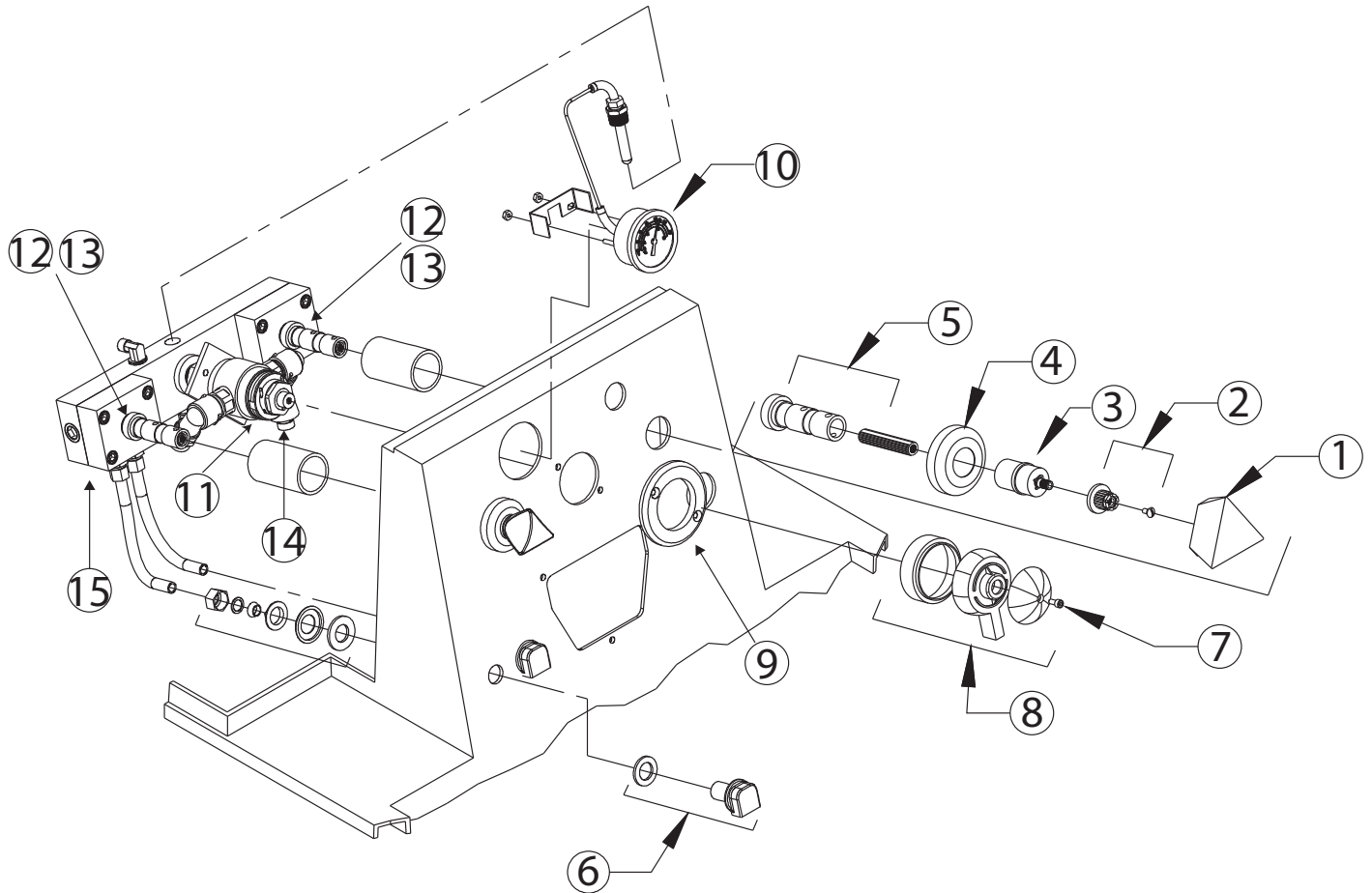

Whirlpool Cleaning/Disinfecting System

Item	Part #	Description	Item	Part #	Description
1	320517	Dual Check Valve (ASME A112.18.3)	7	320319	1/2" Comp x 3/8" MPT 90 Elbow
2	350265	1/2" Hose Clamp	8	320521	Supply Hose .375 Comp x 20"
3	500154	Injector Barb Replacement Kit	9	300087	1/4" ID Poly Tubing 30"
4	401006	Chemical Injector	10	300094	1/4" ID x 3/8" OD Plastic Tubing 6"
5	215115	Whirlpool DS Injector Valve Assy	11	200374	DS Check Valve Assembly
6	320223	Check Valve			

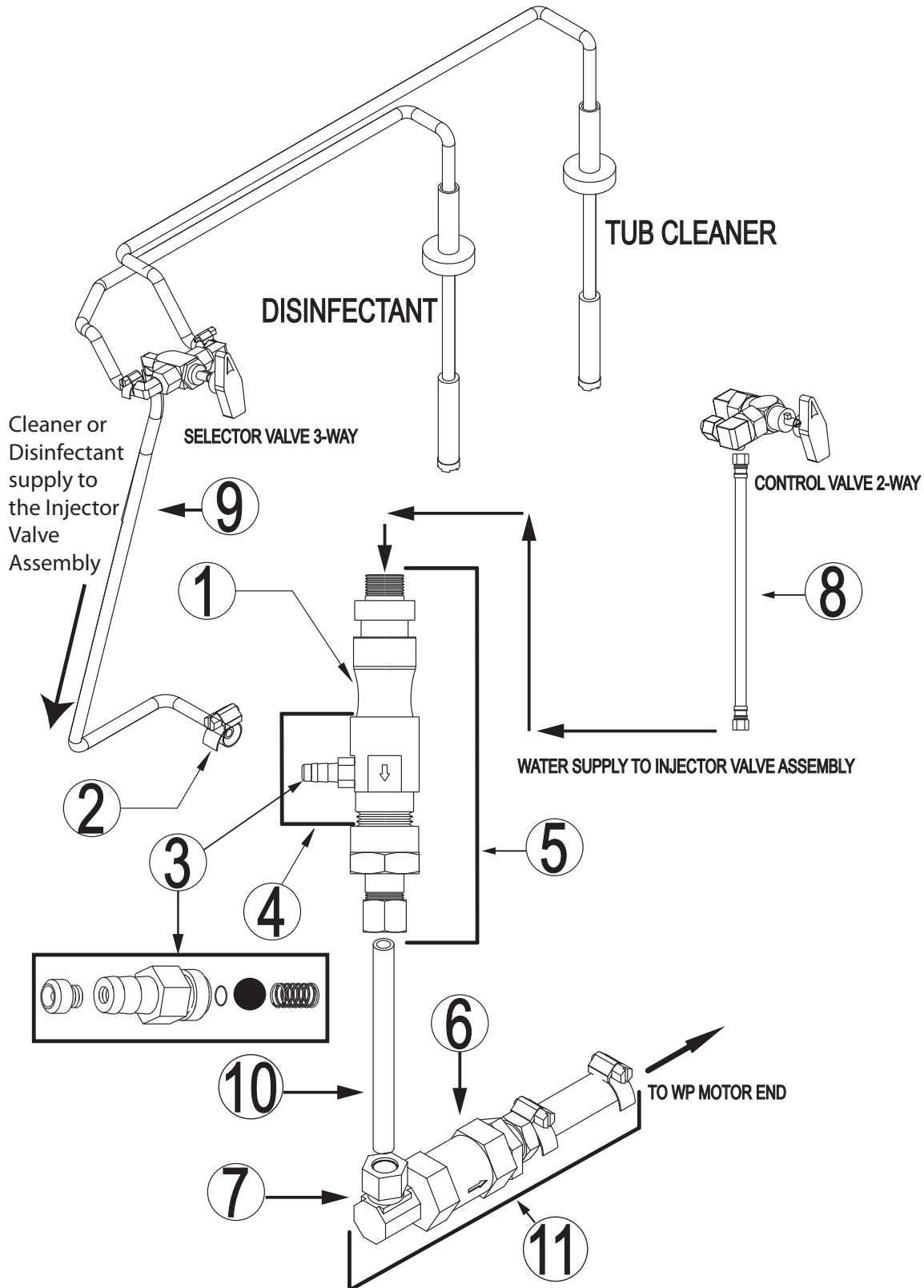


This exploded view diagram illustrates the assembly of a mechanical component, likely a pump or valve. The main body (1) is a complex casting with various ports and mounting points. Key components and their assembly sequence are indicated by the numbered callouts:

- 1**: Main body casting.
- 2**: Small pin or screw used for internal assembly.
- 3**: Small O-ring or seal.
- 4**: Small pin or screw used for internal assembly.
- 5a** and **5b**: Two views of a small rectangular component, possibly a solenoid or actuator, mounted on the side of the main body.
- 6**: Small pin or screw used for internal assembly.
- 7**: Small pin or screw used for internal assembly.
- 8**: Flange or mounting bracket attached to the side of the main body.
- 9**: Small pin or screw used for internal assembly.
- 10**: Long, flexible hose or cable connected to the bottom of the main body.
- 11**: Small O-ring or seal at the bottom connection.
- 12**: Small pin or screw used for internal assembly.
- 13**: Small pin or screw used for internal assembly.
- 14**: Small pin or screw used for internal assembly.
- 15**: Small pin or screw used for internal assembly.
- 16**: Small pin or screw used for internal assembly.
- 17**: Large, flat plate or cover that fits over the top and side of the main body.
- 18**: Small pin or screw used for internal assembly.
- 19**: Small pin or screw used for internal assembly.
- 20**: Small pin or screw used for internal assembly.
- 21**: Small pin or screw used for internal assembly.
- 22**: Small pin or screw used for internal assembly.

Whirlpool Jet Assemblies


Item	Part #	Description
1	200310	Upper Jet Assembly
2	320277	Flange
3	200309	Lower Jet Assembly
4	320278	Jet Eye
5	320276	Jet Escutcheon

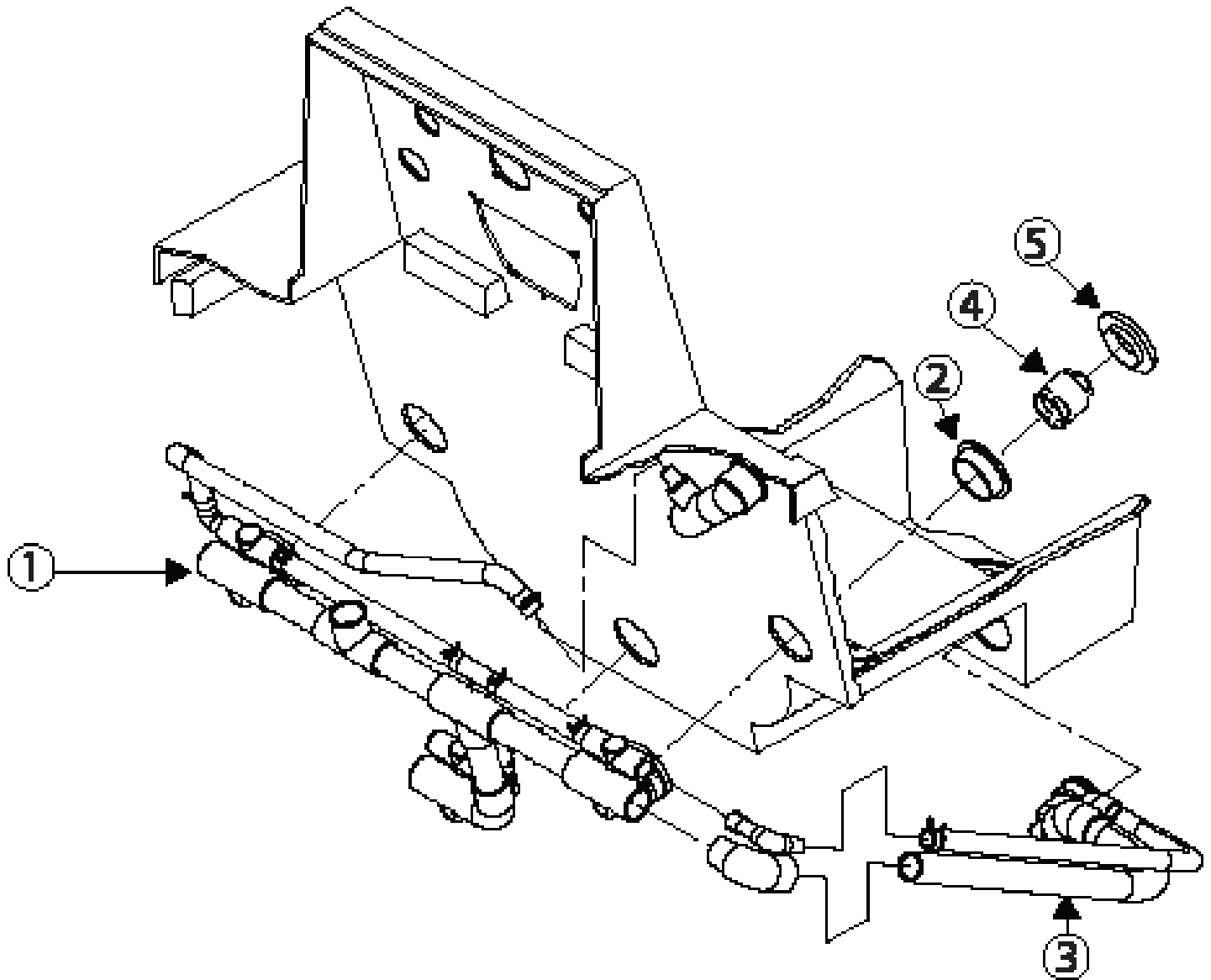


Remedy® UV system


Item	Part #	Description
1	400230	Nipple Male
2	350256	7/8 IDx1 1/8 OD Rubber O-Ring
3	N/A	Quartz Tube Assembly
4	330056	U.V. Lamp 18" Long
5	300049	5/8" ID Plastic Tubing Black
6	200229	Remedy® UV system Indicator
6a	200426	LED Wire Harness Assembly
6b	200427	Solar Cell Assembly
7	500130	Ballast Replacement Kit (1 ballast)
8	330202	LMF Time Delay Fuse .5 Amp
9	330201	LHR Fuse Holder

Item	Part #	Description
10		Hanger
11	350244	8-32 x 3/4" PH SSMS
12	500151	Digital Ballast Upgrade Kit
Kit	500157	Quartz Tube Replacement Kit - includes items 2 and 3

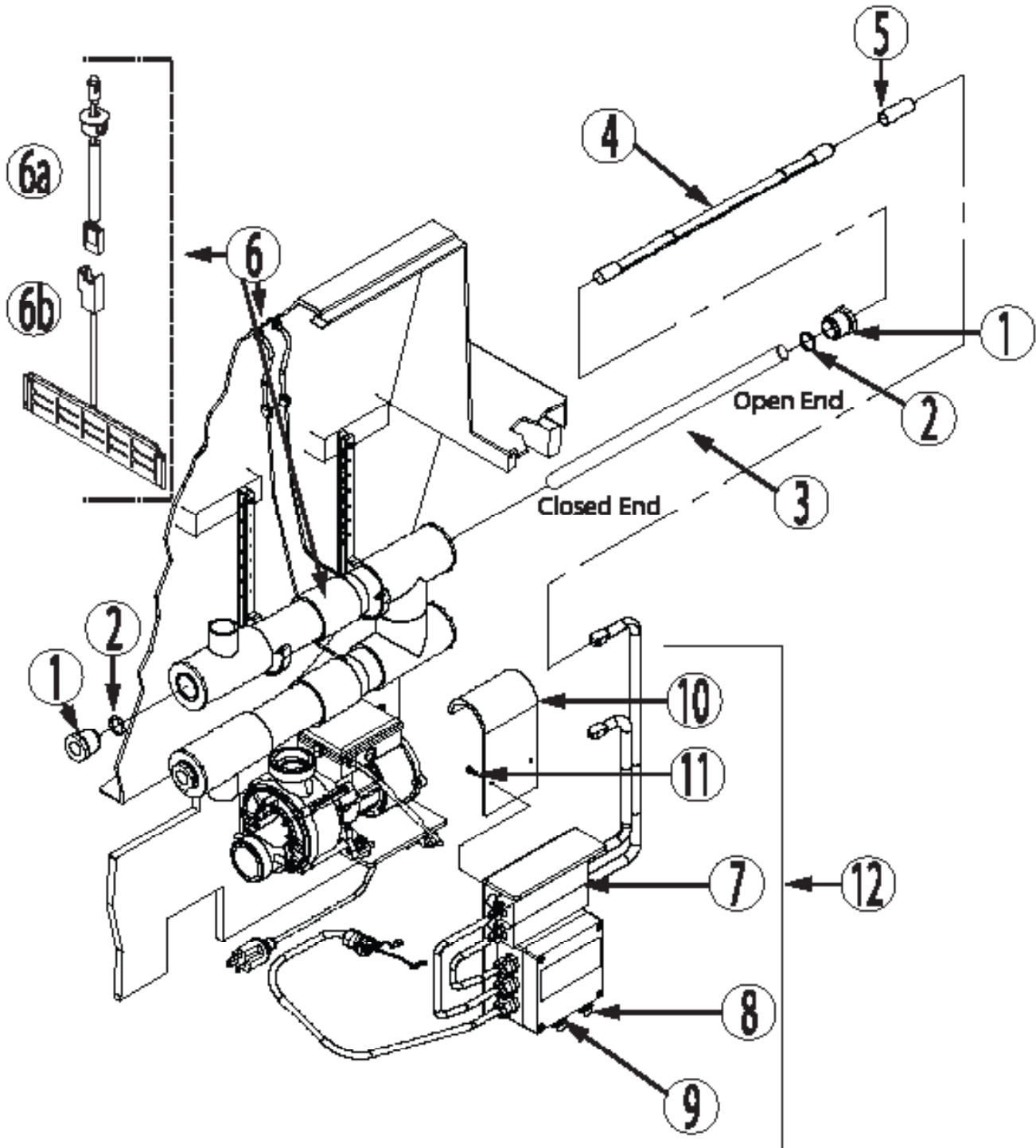

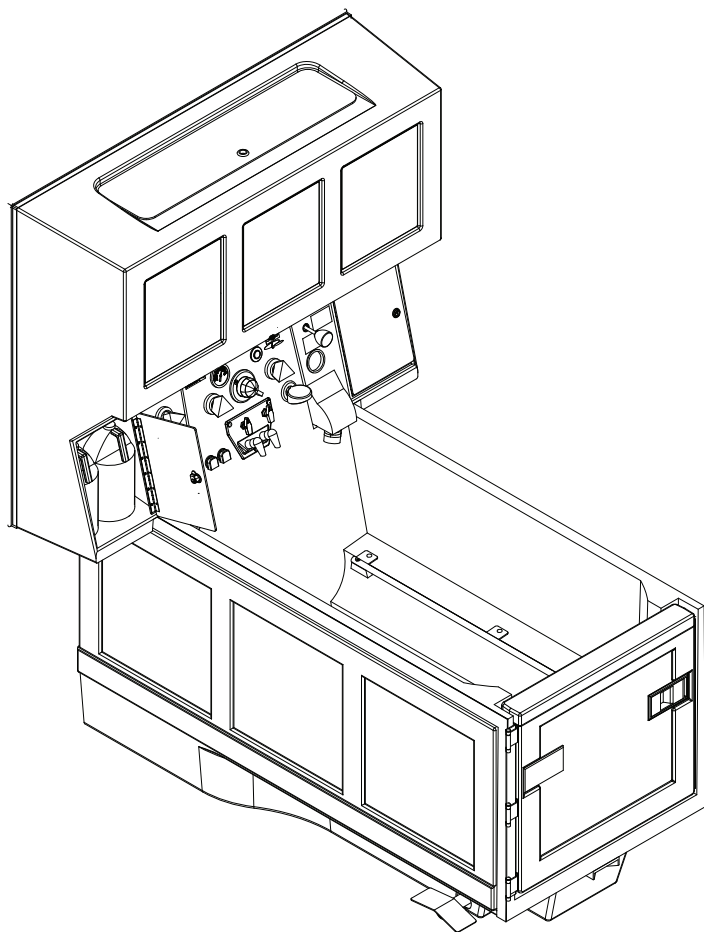


6000 Series

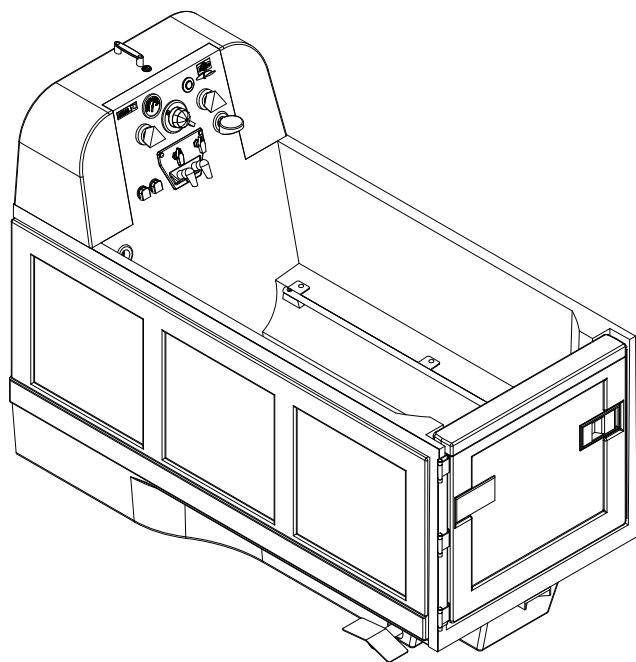

6300 Series


Air Spa Control Panel (2009 - 2014)

Item	Part #	Description	Item	Part #	Description
1	320329	Chrome Knob	11	510082	Valve Cartridge 1/2" 90 Degree
2	510087	Valve Cartridge 3/4" 90 degree	12	510084	Valve Seat 1/2"
3	510088	Renewable Seat Set 3/4"	13	320420	Fill Spouts
4	320295	3-Way Valve	14	320296	2-Way Valve
4	200317	3-Way Valve with 90 degree elbows	14	200313	2-Way Valve with 90 degree elbows
5	510066	Rada 222 Control Knob	15	200440	Shampoo/Body Wash Pump
6	510071	Rada 222 Radatherm Cartridge	16	200441	Bath Oil Pump
7	350336	Thermometer	17	400856	Solutions Panel
8	380037	Key Pad	Kit	500147	Skin Care Pump Kit (items 15 & 16)
9	320005	Shower Hose			
10	320006	Shower Wand			

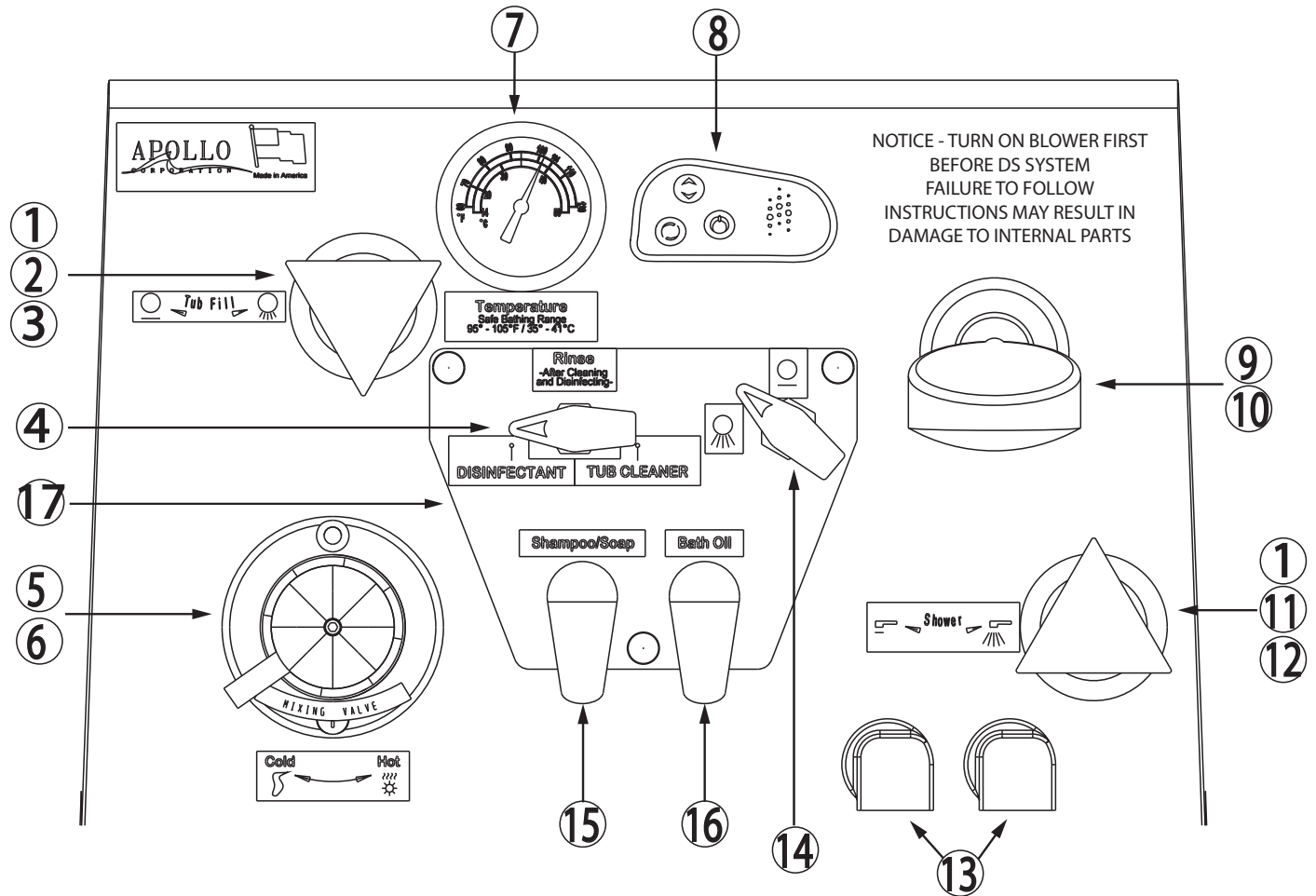


Air Spa Control Panel (2014 - Present)

Item	Part #	Description	Item	Part #	Description
1	320329	Chrome Knob	11	320420	Fill Spouts
2	510087	Valve Cartridge 3/4" 90 degree	12	320296	2-Way Valve
3	510088	Renewable Seat Set 3/4"	12	200313	2-Way Valve with 90 degree elbows
4	320522	3-Way Valve	13	200440	Shampoo/Body Wash Pump
	200435	3-Way Valve with 90 degree elbows	14	200441	Bath Oil Pump
5	510066	Rada 222 Control Knob	15	400856	Solutions Panel
6	510071	Rada 222 Radatherm Cartridge	Kit	500147	Skin Care Pump Kit (items 13 & 14)
7	350336	Thermometer			
8	380037	Key Pad			
9	320005	Shower Hose			
10	320006	Shower Wand			

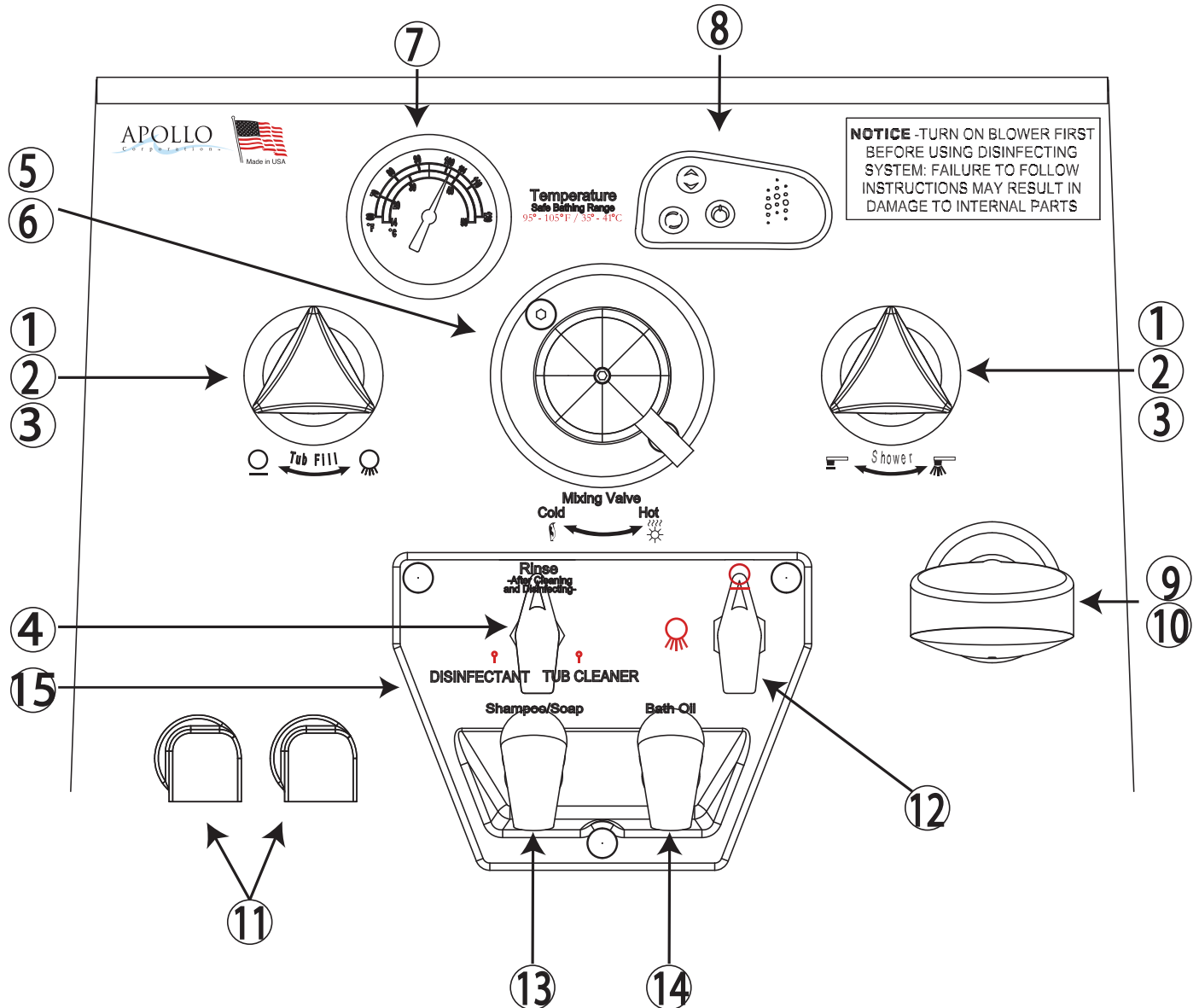


Air Spa Control Panel Valves (2009 - 2014)

Item	Part #	Description	Item	Part #	Description
1	320329	Chrome Knob	9	450073	Rada Deck Plate 222
2	510090	Handle Connection Set	10	350336	Thermometer
3	510091	Extension Sleeve	11	510071	Rada 222 Radatherm Cartridge
4	320423	Classic Flange	12	510082	Valve Cartridge 1/2" 90 Degree
5	510086	Extension Spindle	13	510084	Valve Seat 1/2"
6	320420	Fill Spouts	14	510087	Valve Cartridge 3/4" 90 degree
7	510067	Handle Locking Screw Rada 222	15	510088	Renewable Seat Set 3/4"
8	510066	Rada 222 Control Knob	16	320003C	Vacuum Breaker (ASSE 1014)

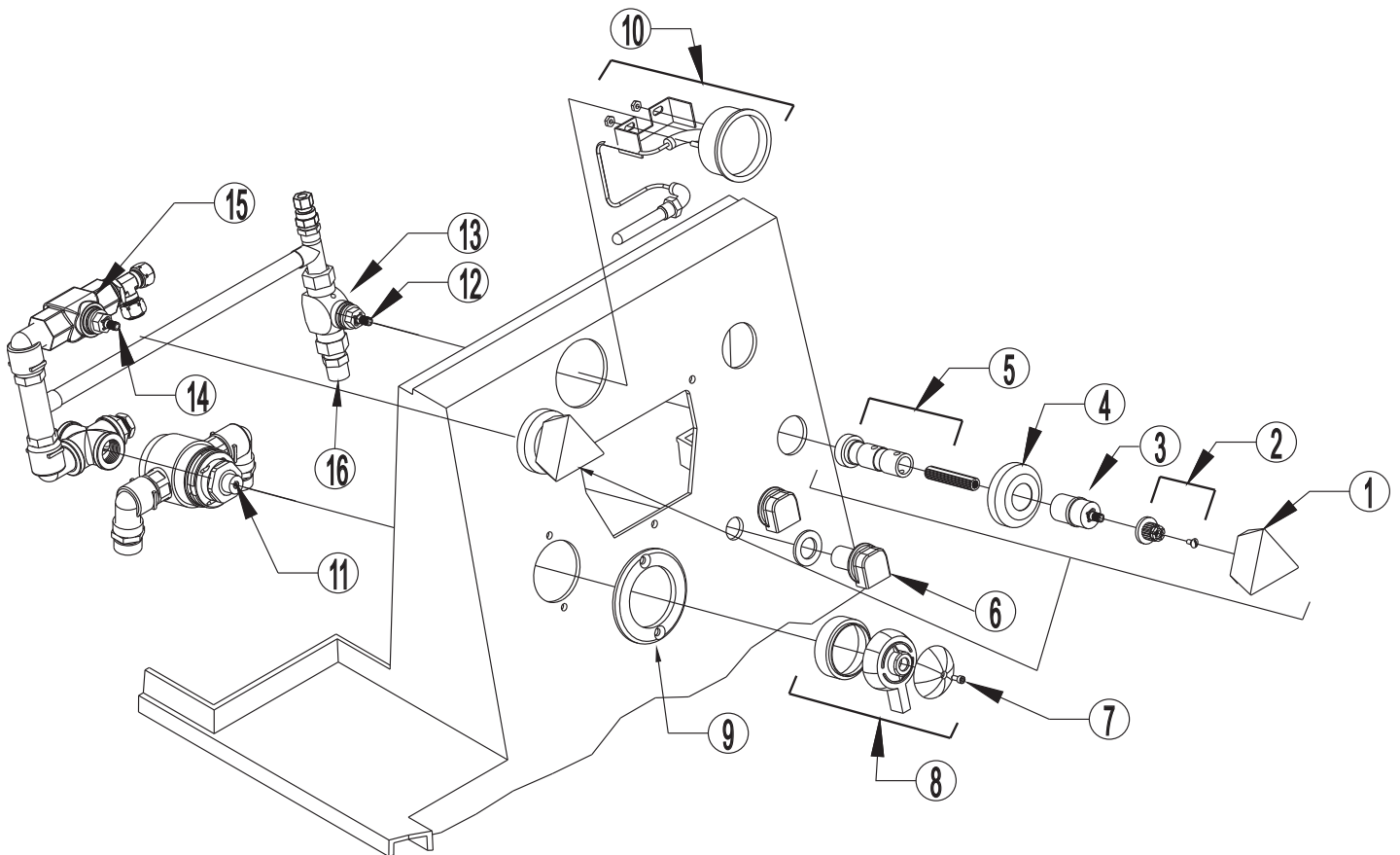


Air Spa Control Panel Valves (2014 - Present)

Item	Part #	Description	Item	Part #	Description
1	320329	Chrome Knob	9	450073	Rada Deck Plate 222
2	510090	Handle Connection Set	10	350336	Thermometer
3	510091	Extension Sleeve	11	510071	Rada 222 Radatherm Cartridge
4	320423	Classic Flange	12	510087	Valve Cartridge 3/4" 90 degree
5	510086	Extension Spindle	13	510088	Renewable Seat Set 3/4"
6	320420	Fill Spouts	14	320003C	Vacuum Breaker (ASSE 1014)
7	510067	Handle Locking Screw Rada 222	15	500352	Water Manifold Assembly
8	510066	Rada 222 Control Knob			

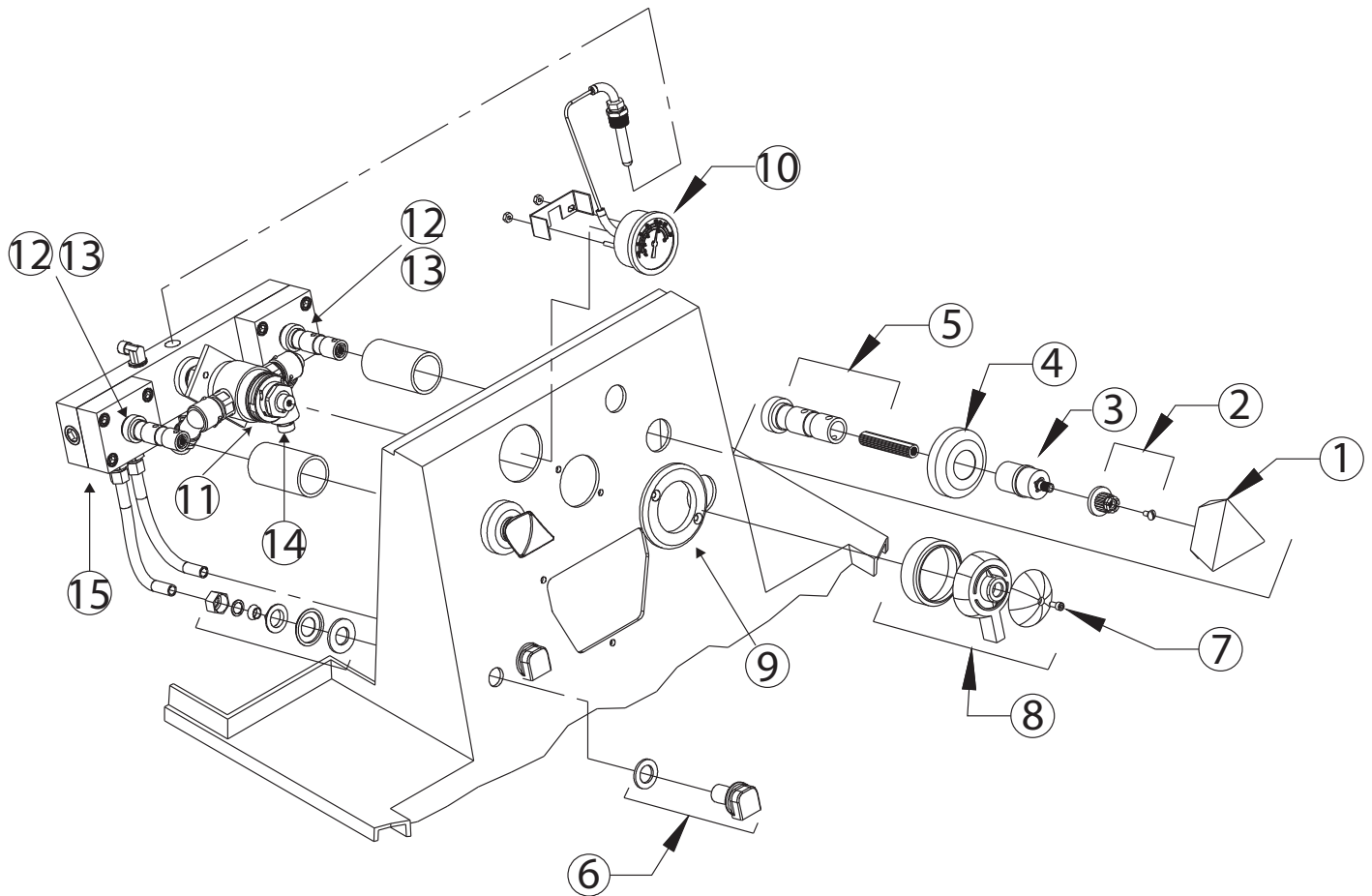


Air Spa Cleaning/Disinfecting System

Item	Part #	Description	Item	Part #	Description
1	320517	Dual Check Valve (ASME A112.18.3)	7	320319	1/2" Comp x 3/8" MPT 90 Elbow
2	350265	1/2" Hose Clamp	8	320521	Supply Hose .375 Comp x 20"
3	500154	Injector Barb Replacement Kit	9	300087	1/4" ID Poly Tubing 30"
4	401006	Chemical Injector	10	300094	1/4" ID x 3/8" OD Plastic Tubing 6"
5	215160	Air Spa DS Injector Valve Assy	11	200374	DS Check Valve Assembly
6	320223	Check Valve			

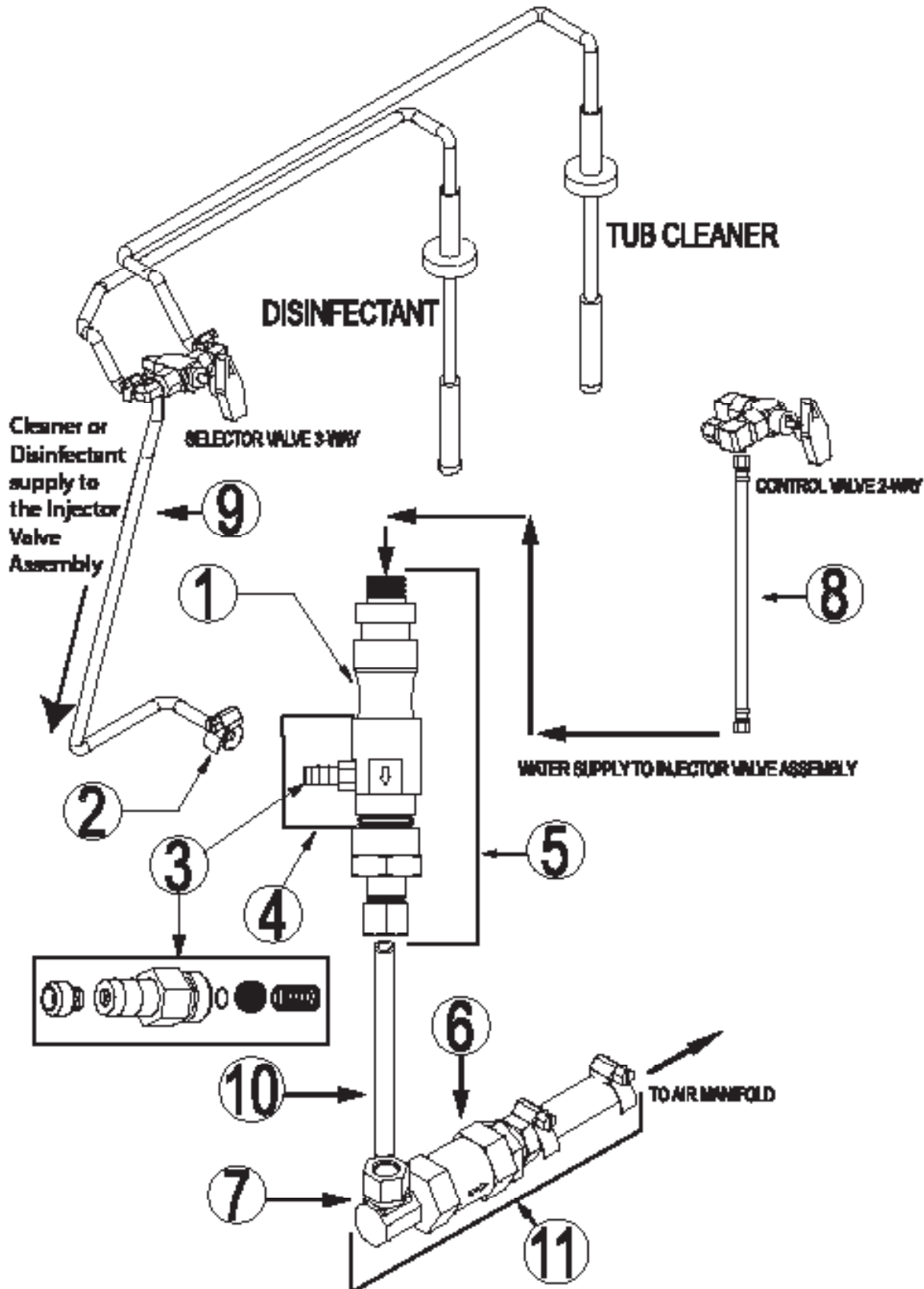


Air Spa Blower & Manifold


Item	Part #	Description	Item	Part #	Description
1	200323	Air Manifold Assembly	7	380044	6' Hose w/Clamps
2	350027	10-32 x 1 1/2" Std SS HHMS			
3	350229	#10 Flat Washer SS			
4	350011	10-32 Nylock Hex Nut SS			
5	380036	Tms Contr Var Blower			
6	500137	Replacement Blower Kit			

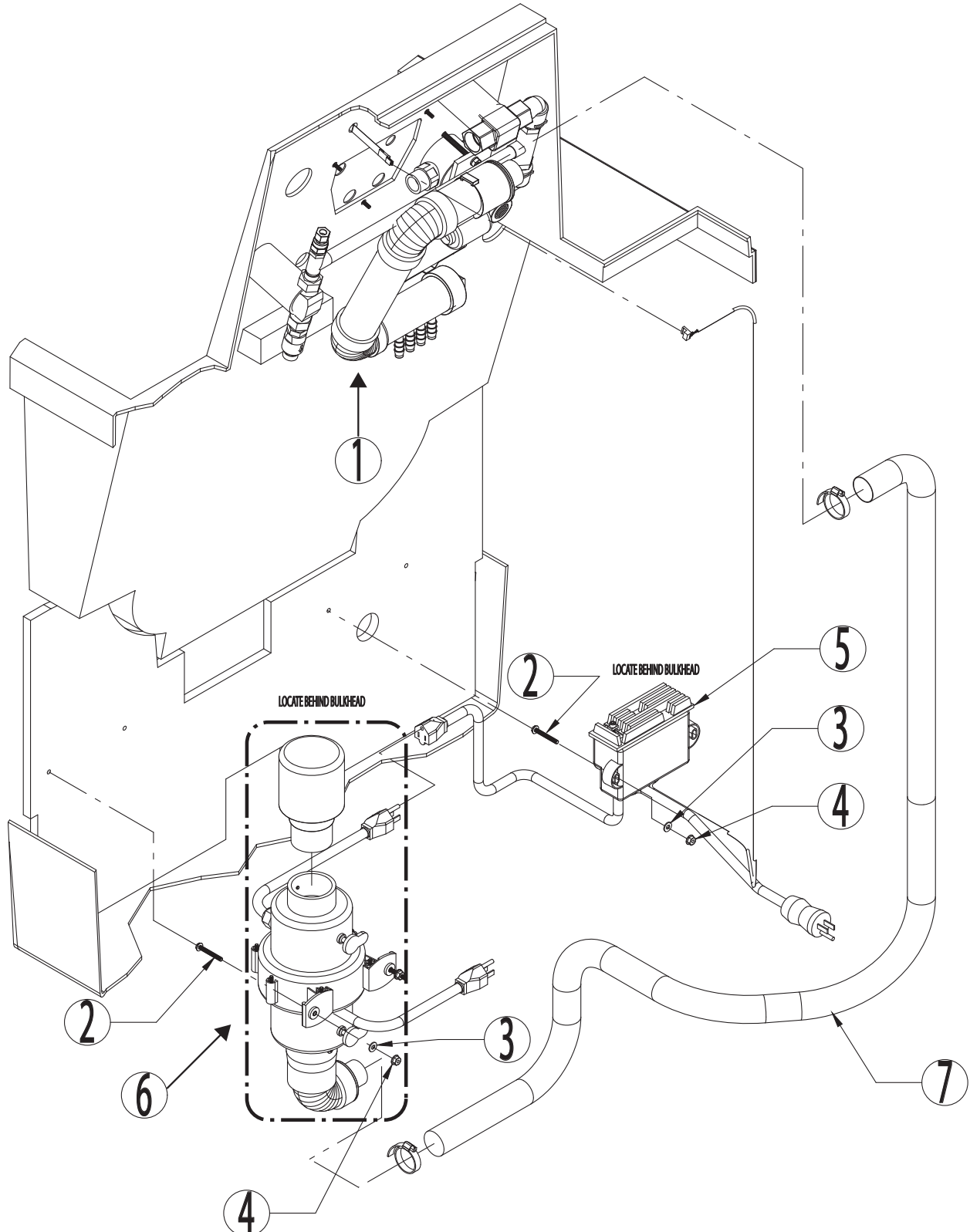


Air Spa Full Size / Space Saver System

Item	Part #	Description
1	380033	Air Jet Multi-Hole
1a	380058	Air Jet Replacement Cap Multi-Hole
2	380038	Jet PVC Elbow 1 Outlet + Washer
3	380039	Jet PVC Elbow 2 Outlet + Washer
4	380034	Air Jet No Holes
4a	380059	Air Jet Replacement Cap No Holes
5	300177	3/8" Flexible Hose (Sold by the foot)

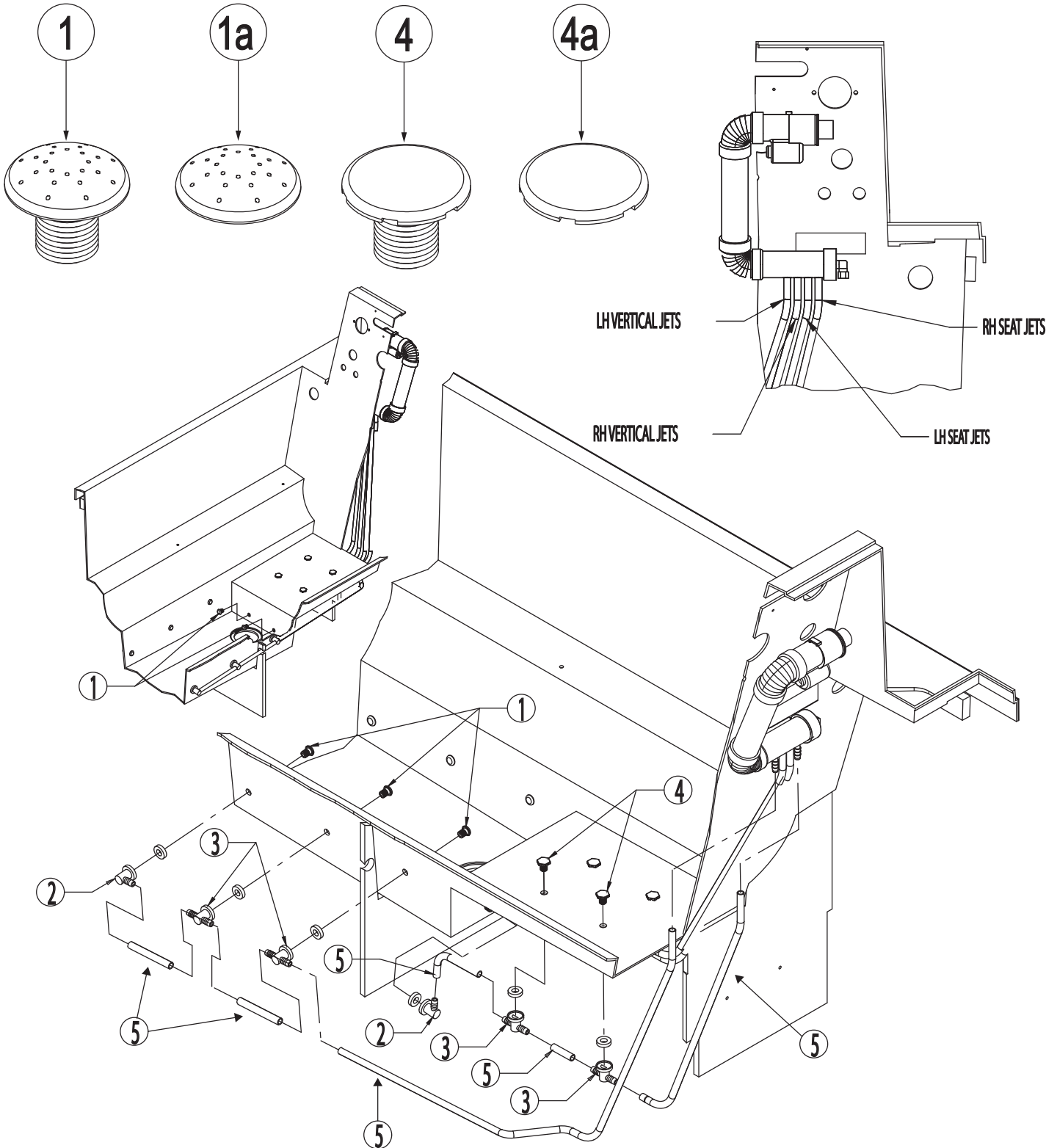

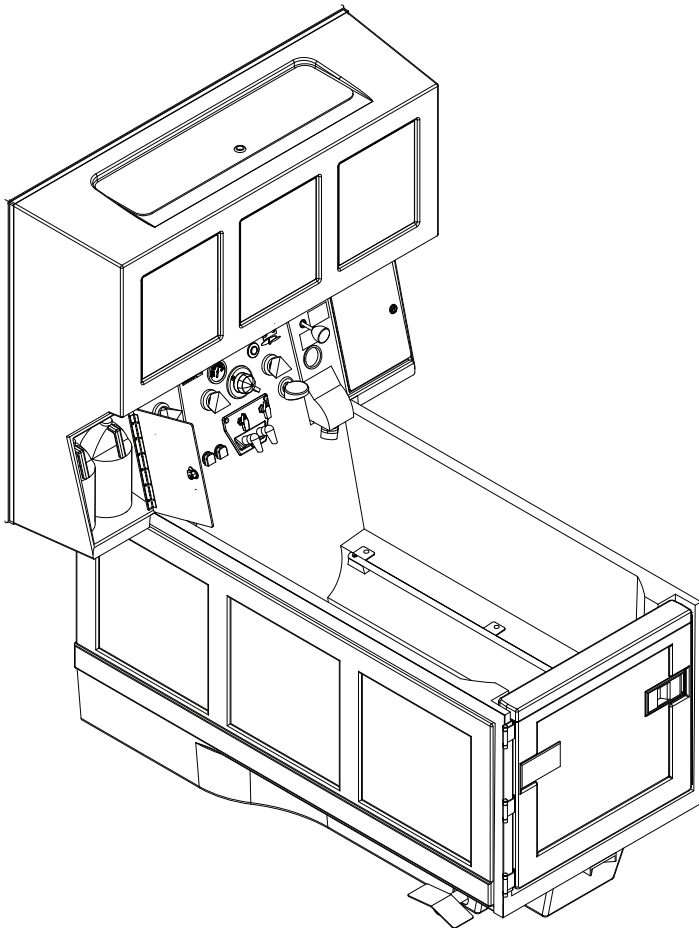


WP6000 Series

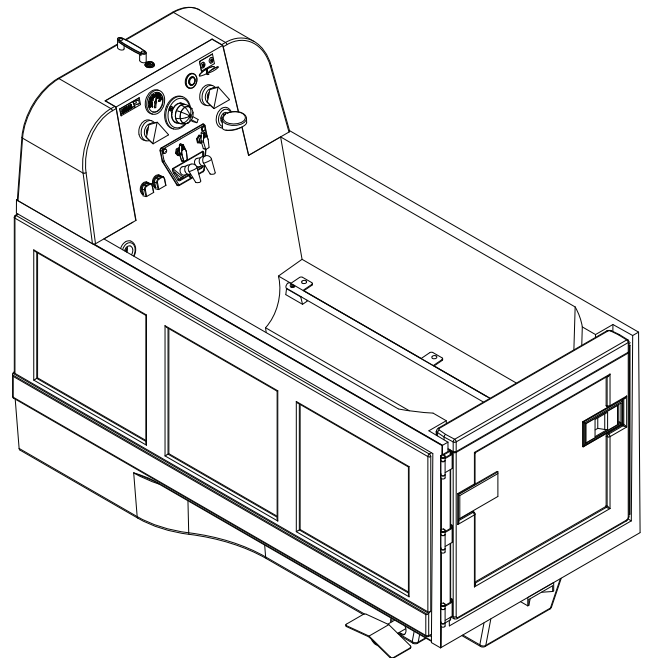

WP6300 Series


Tub Assembly

Item	Part #	Description	Item	Part #	Description
1	350560	Grabber Catch w/ Heavy Duty Keeper	10	340422	Rail Stop
2	350225	#4 Washer	11	350078	1/4-20 Nylon Insert Hex Nut SS
3	350072	4-24 x 1/2" Std PPHSMS	12	200261	Tub Safety Strap-Tubular
4	350284	1/4-20 UNC x 1 Bolt	13	350156	1/4-20 x 1" Machine Screw SS
5	200301	Side Panel Assy	14	350161	1/4" Flat Washer SS
6	200385	Space Saver Side Panel Assy	15	350155	1/4" Lock Washer
7	400363	Tub Rail 46.75 in (2005-2012)	16	350044	1/4-20 UNC Std Hex Nut SS
7	400980	Tub Rail 48.25 in (2013-Present)	17	300167	Vinyl Rounded Trim Molding
8	400364	Space Saver Tub Rail 34.75 in (2005-2012)	18	340010	Receiver Block
8	400981	Space Saver Tub Rail 36.25 in (2013-Present)	19	400871	Check Valve
9	350381	1/4-20 x 1 3/4" Bolt SS			

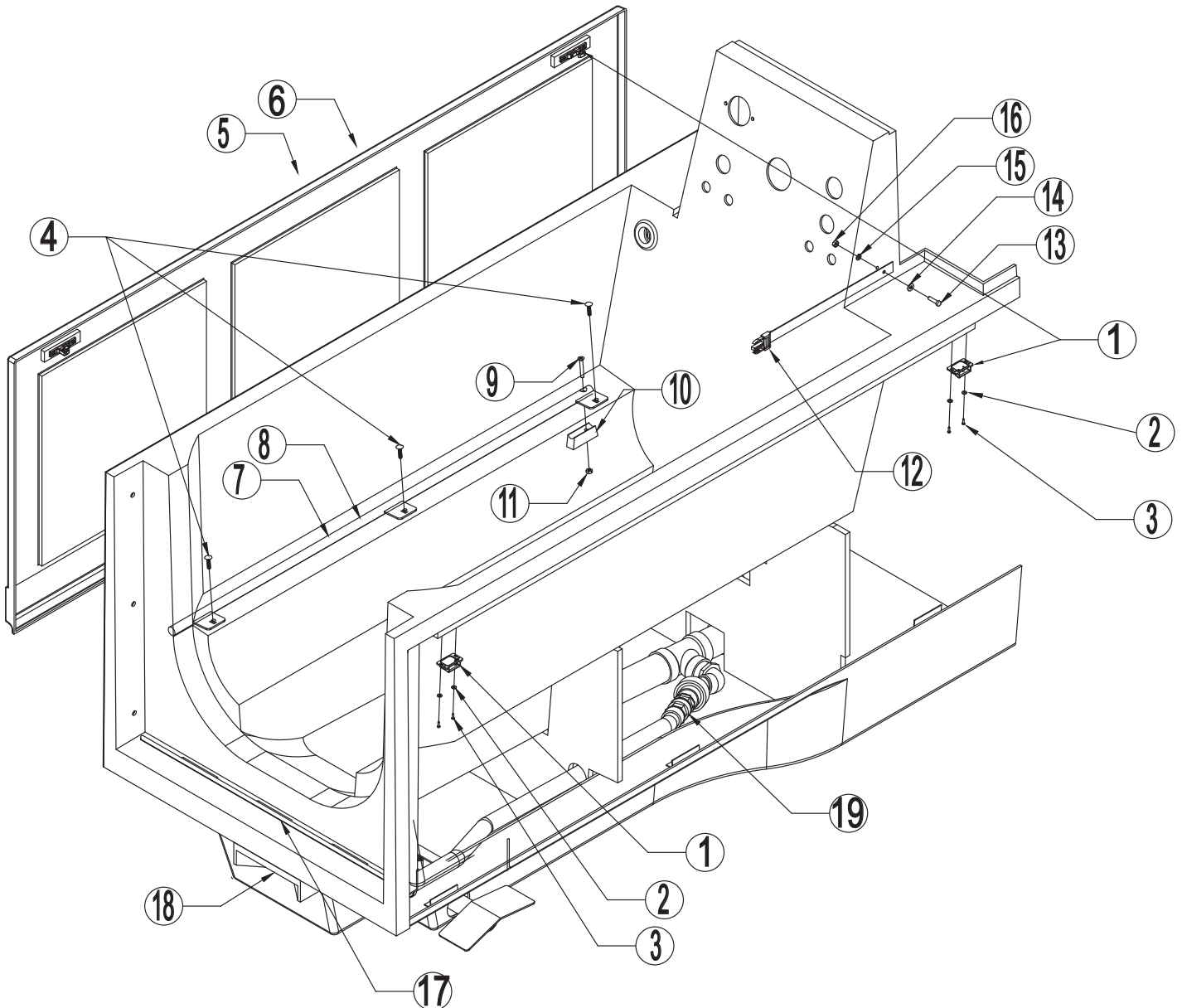


Tub Door

Item	Part #	Description	Item	Part #	Description
1	400790	Door Inner w/Metal Frame Assy	12	350576	10-24 x 3/4" CRFCSHMS SS
2	400773	Door Hinge Plate	13	350574	10-24 Spring Clip U-Nut
3	350554	10-24 x 1/2" Flat Head Screw - SS	14	350575	.50 x .093 Vibration Washer
4	350551	Remote Compression Latch	15	400865	Door Outer Skin (includes 350575)
5	350553	10-24 x 1/2" Pan Head Screw - SS	16	350229	10 Flat Washer SS
6	350311	10 STD, ETLW SS	17	500105	Door Latch Hardware (1 set)
7	300175	Hexagonal Rod	18	350577	Door Hinge Pin 1/4 x 3" SS
8	350550	Paddle Handle Actuator	19	500104	Door Gasket Replacement Kit
9	400711	Door Handle Plug	20	500122	Door Latch Replacement Kit (3 sets)
10	400769	Door Plug Retainer			
11	350570	8-32 x 3/8" CRPHSMS - SS			

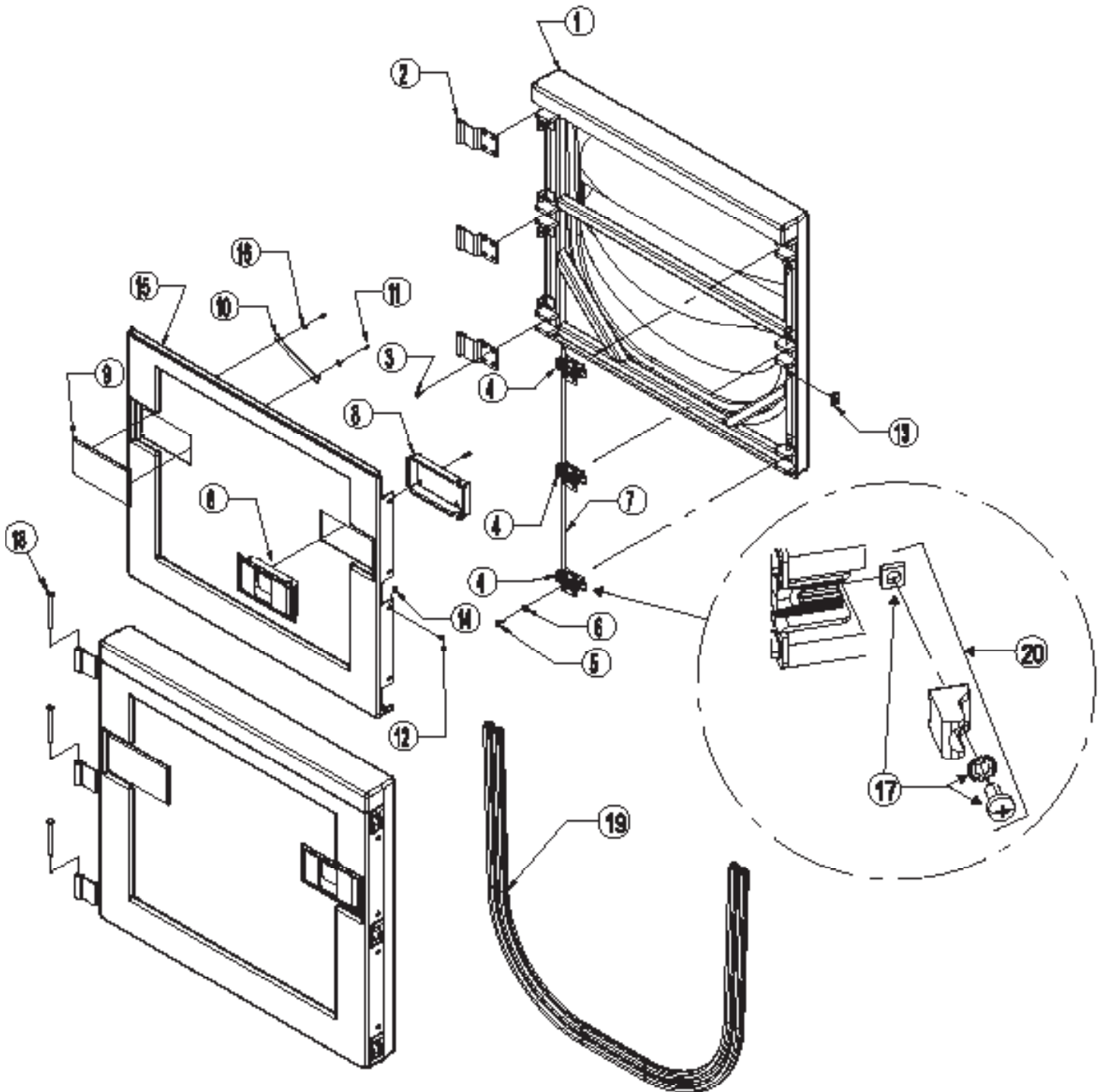

Tub Drain Assembly


Item	Part #	Description
1	320263	Chrome Flange and Screen
2	200148	Slide Valve Drain Assembly
3	350233	1/4 ID x 3/8 OD x 1/4 Nylon Bushing
4	350232	1/4" Shaft Retainer Ring
5	400768	Drain Lever Arm Stop
6	350161	1/4" Flat Washer SS
7	350619	RHPM Screw #10-32 x 1"; SS
8	350011	10-32 NF Nylock Nut SS
9	340010	Receiver
10	350078	1/4-20 Nylon Insert Hex Nut SS
11	350355	1/4-20 x 1" PHIL TRUSS HD Bolt SS

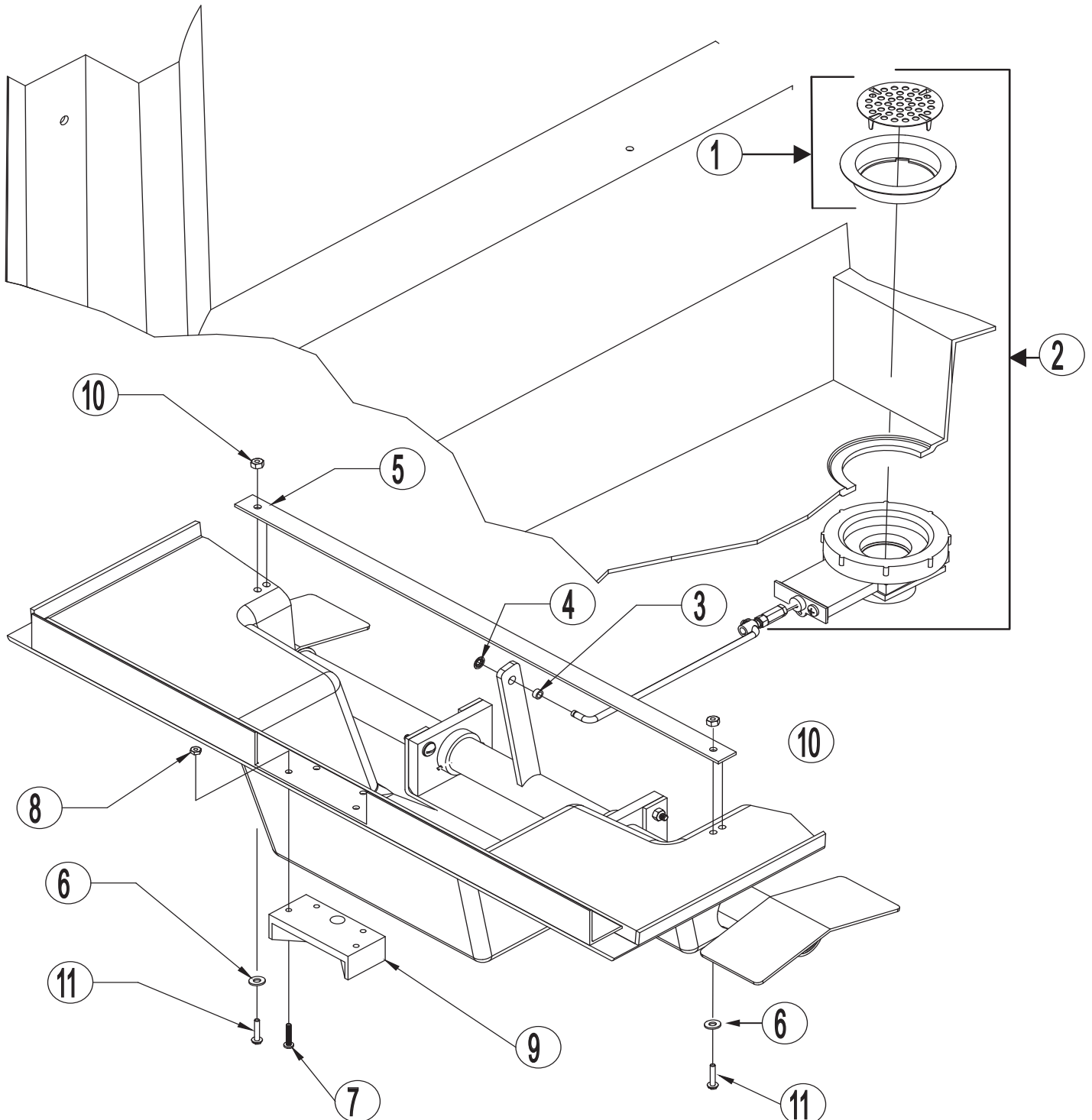

Slide Valve Drain Assembly - 200344


Item	Part #	Description
1	320134	Drain Adaptor
2	410166	2" Slide Valve Body
3	350229	10 Flat Washer SS
4	350027	10-32UNF x 1 1/2" Std SS HHMS
5	320260	Slide Valve Flange (Male)
6	350183	1/4-20 Jam Nut SS
7	350196	1/4-20 x 7/8" Coupling Nut SS
8	350598	Ball Joint Linkage
9	350232	1/4" Shaft Retainer Ring
10	400812	90 Deg x 90 Deg Drain Actuator Rod

Item	Part #	Description
11	320313	2"x 90 Deg Pressure Elbow PVC
12	300031	2" Dia Rigid PVC
13	320519	Fernco Coupling 2" x 2"

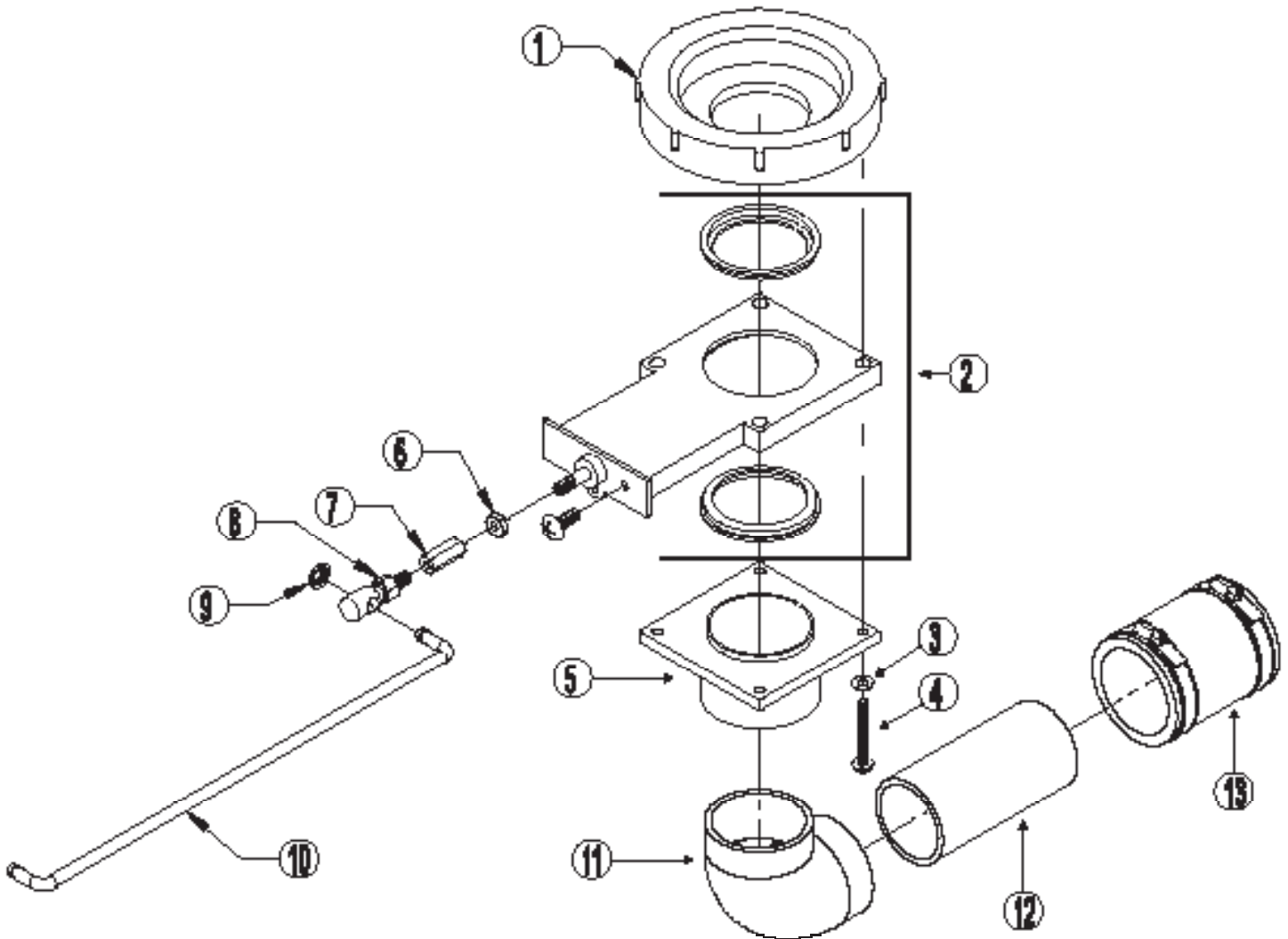


Tub Shroud Assembly - 200348

Item	Part #	Description
1	400710	Tub Shroud
2	350506	Cabinet Lock
3	350601	Shroud Handle
4	350078	1/4-20 Nylon Insert Hex Nut SS
5	350039	1/4-20 x 1 1/2" CRPHMS SS
	350005	Cabinet Keys (not shown)

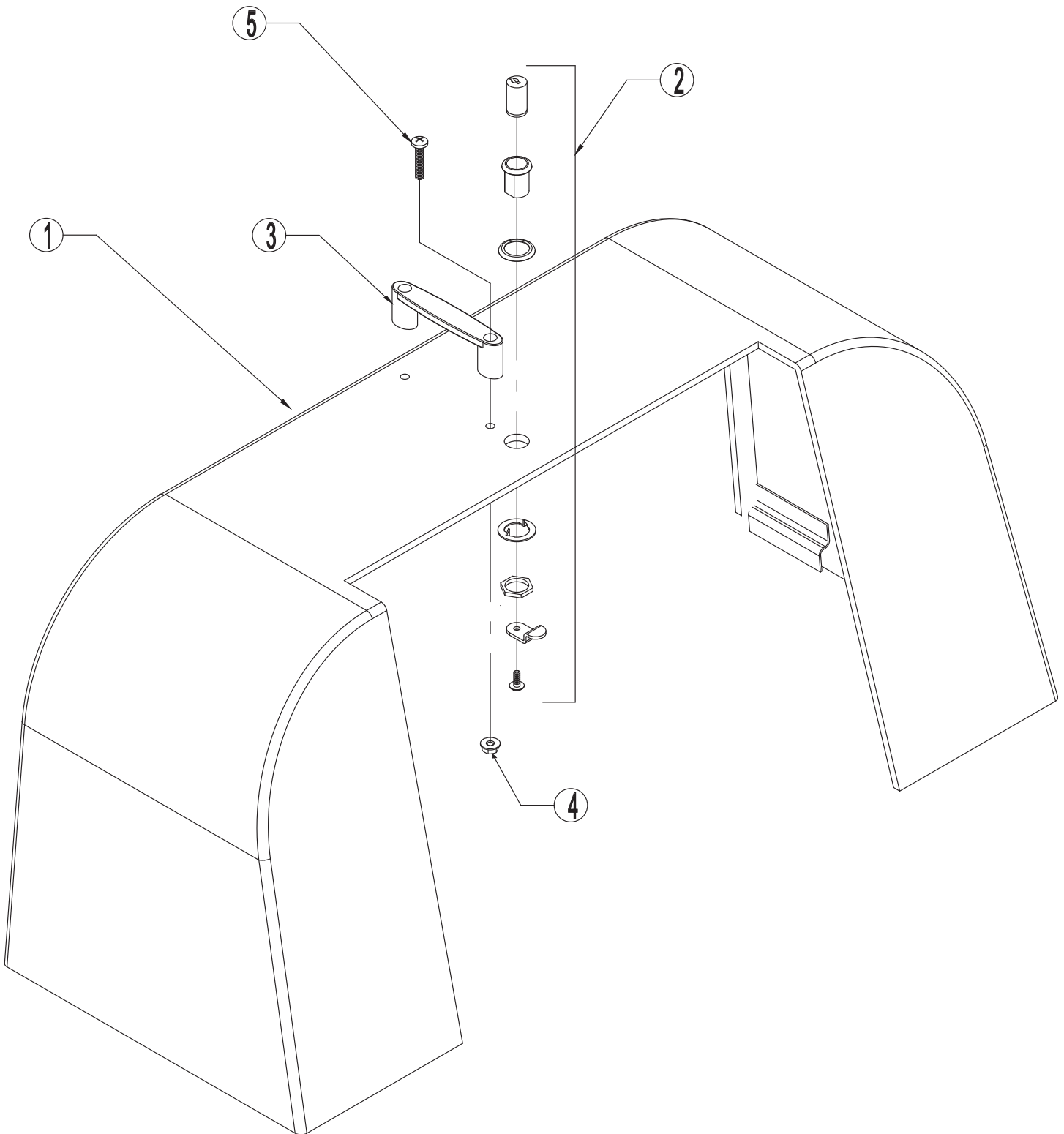


Reservoir Cabinet Shroud

Item	Part #	Description	Item	Part #	Description
1	210067	RH Cabinet Door Assy	12	410091	Cabinet Top Panel
2	210068	LH Cabinet Door Assy	13	340250	Top Panel Hole Grommet Cab
3	350336	Thermometer	14	300152	Cabinet Trim Seal
4	410182	Cabinet Slide Valve Escutcheon	15	510082	Valve Cartridge 1/2" 90 Degree Right
5	410184	Slide Valve Extension Rod	16	510084	Valve Seat 1/2"
6	340154	Valve Knob	17	320326	1/2" Ferrule Plastic
7	510086	Valve Extension Spindle	18	300135	3/8 ID x 1/2 OD PEX Tubing
8	320423	Valve Flange	19	210070	Tub Fill Stop (Nut & Washer)
9	510091	Extension Sleeve	20	320489	Supply Hose; SS .50 x 60"
10	510090	Handle Connection	21	210050	Reservoir Tank Assembly
11	320329	Chrome Knob			

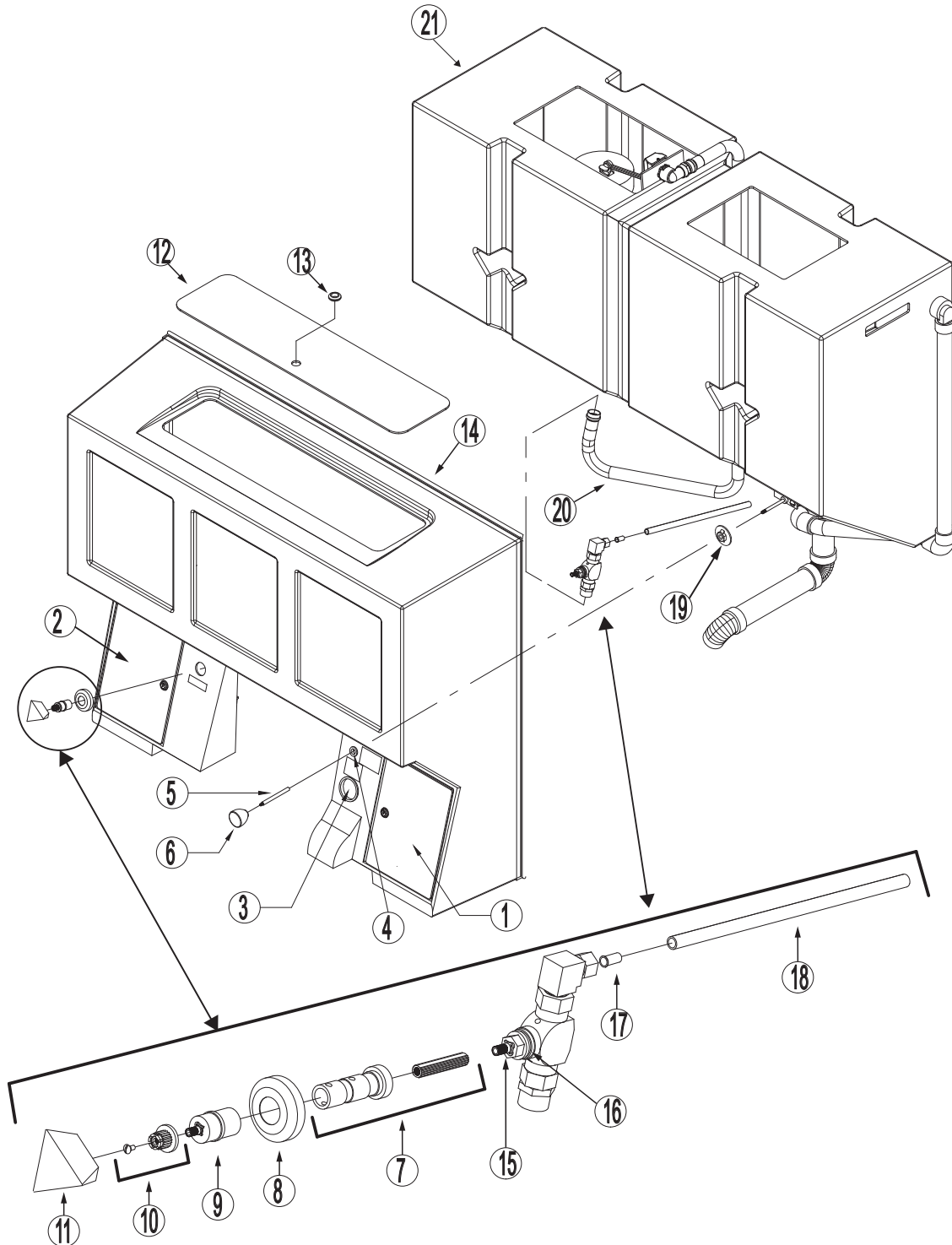

Reservoir Tank Assembly - 210050


Item	Part #	Description	Item	Part #	Description
1	320489	Supply Hose; SS .50 x 60"	12	320260	Slide Valve Flange (Male)
2	410170	Float Valve Mount	13	350183	1/4-20 Jam Nut SS
3	320493	Float Valve	14	350572	10-24 x 1 1/4" SHCS SS
4	320494	Stainless Steel Rod 6" x .25	15	210057	Reservoir Drain Assy
5	320495	Float Ball 4" x 5"	16	210069	Reservoir Drain Assy Earthquake Models Only
6	350561	10-32 x 1" Hex Washer HMS	17	510093	Replacement Float Valve Kit
7	350229	#10 Flat Washer SS	18	210070	Tub Fill Stop (Nut & Washer)
8	350011	10-32 NF Nylock Nut SS	19	350688	Gasket
9	320225	Over Flow Flange			
10	210051	Reservoir Overflow Assy			
11	410166	2" Slide Valve Body			

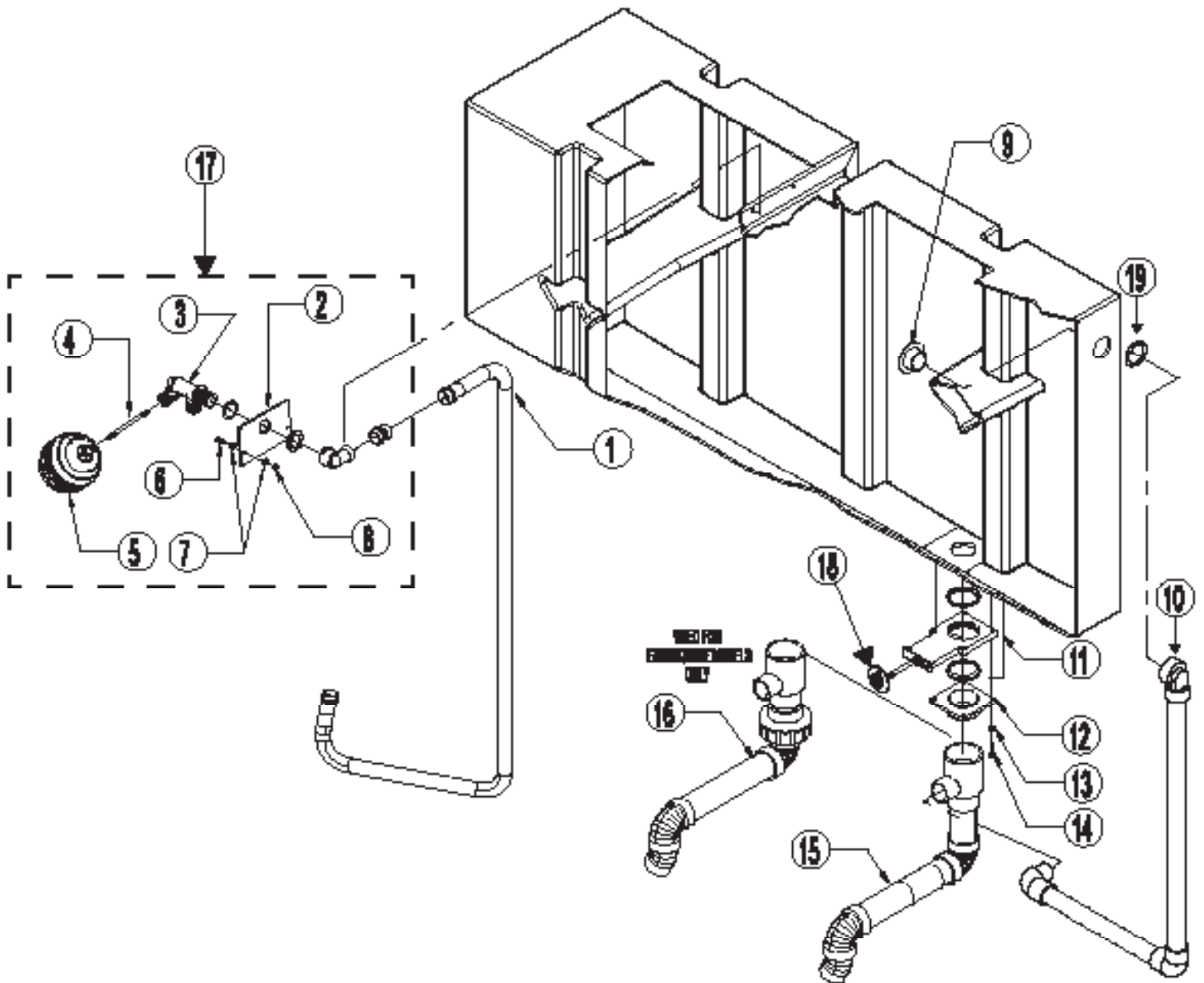

Level Glide™ Transfer System

Note:

For proper part fitment, match the model number on the label attached to the framework of the carrier, as shown in the picture below, with the applicable parts page.

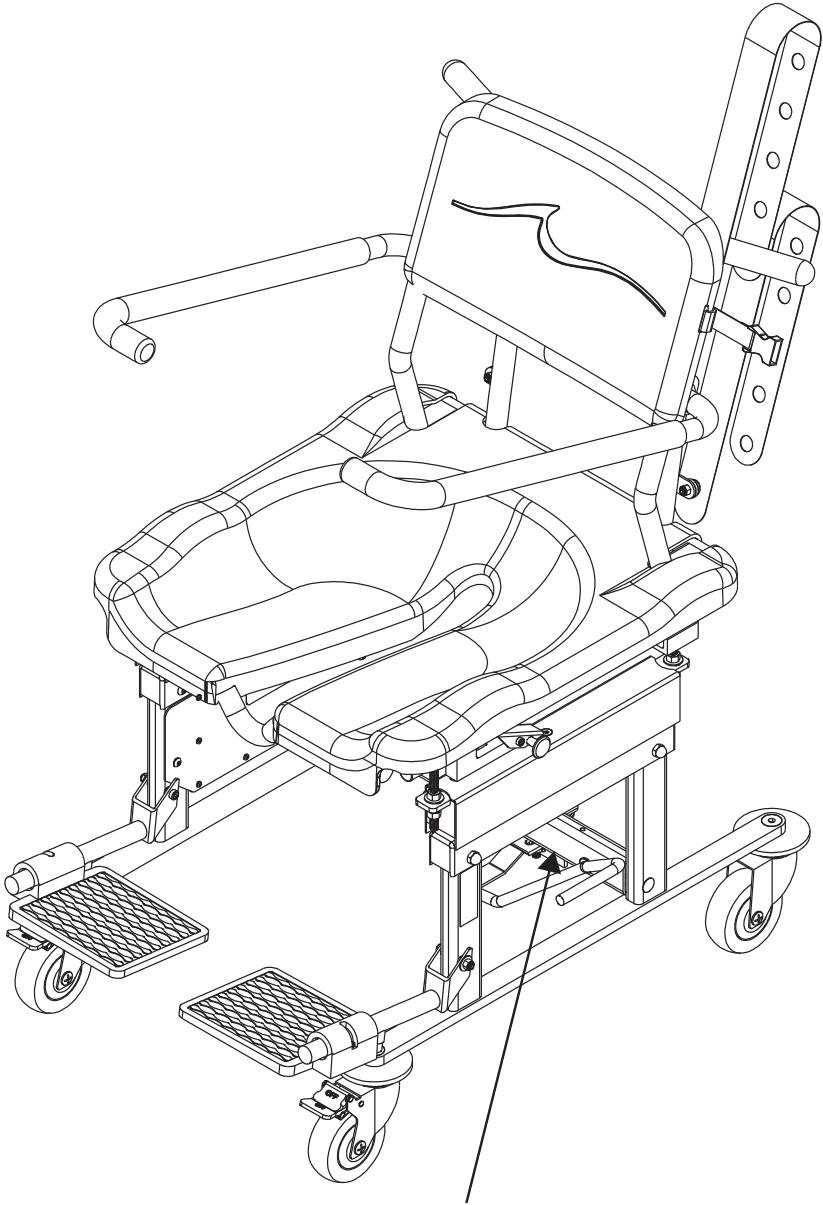

APOLLO CORP.

Somerset, WI 54025


Model No: XXXXXXXXXXXX

Serial No: XXXXXXXXXXXXXX

Date: mm/dd/yyyy

Deluxe Chair Assembly - 220044


Item	Part #	Description
1	420156	Tubular Seat Frame
2	350279	L.R. Wheel
3	350515	1/2" x 1 1/2" Shoulder Bolt SS
4	350518	3/8"-16 Nylock Jam Nut SS
5	350525	1/4-20 x 1 1/4" HSHCS SS
6	420151	1/4-20 UNC Nylon Knob
7	420154	2" x 72" Seat Belt White
8	340503	SS Chair Back - Black
9	340504	SS Chair Bottom - Black
10	350341	.516 ID x .873 OD x .030 Flat Washer
11	350078	1/4-20 Nylon Insert Hex Nut SS

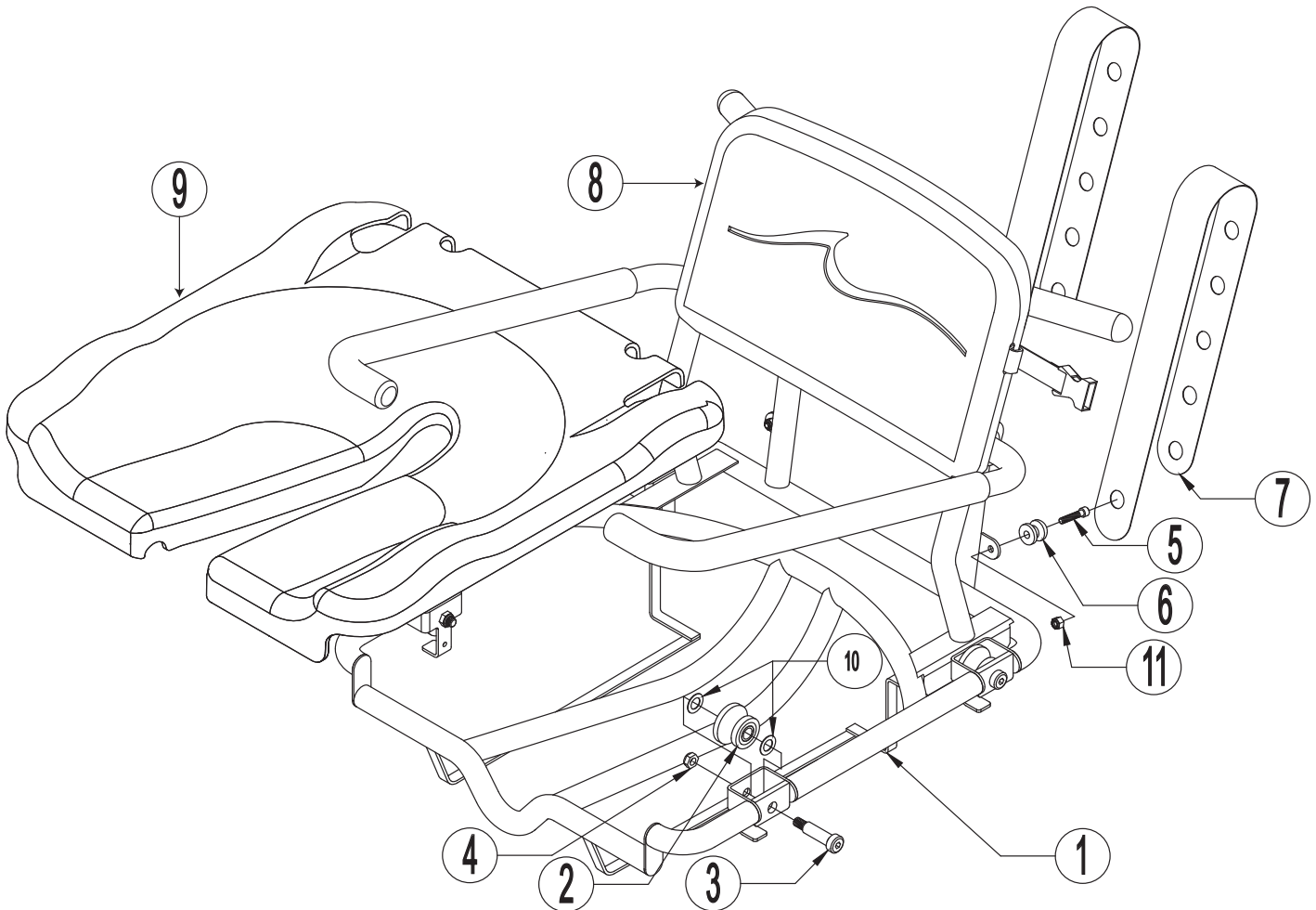

Deluxe Chair Assembly - 220044

Item	Part #	Description
1	420151	1/4-20 UNC Nylon Knob
2	350078	1/4-20 Nylon Insert Hex Nut SS
3	340526	.940 ID x 5" Foam Grip
4	220045	Chair Safety Strap
5	420178	Deluxe Seat Back Strap
6	350620	HSHCS .25-20 x 1.50" SS
7	350538	1" x 1 5/8" x 5/8" Set Collar
8	350539	1" x 1 1/2" x 1/16" Alum Shim
9	340501	Arm Rest Bushing
10	340474	Arm Rest Cover
11	420177	Standard Arm Rest

Item	Part #	Description
12	340503	SS Chair Back - Black

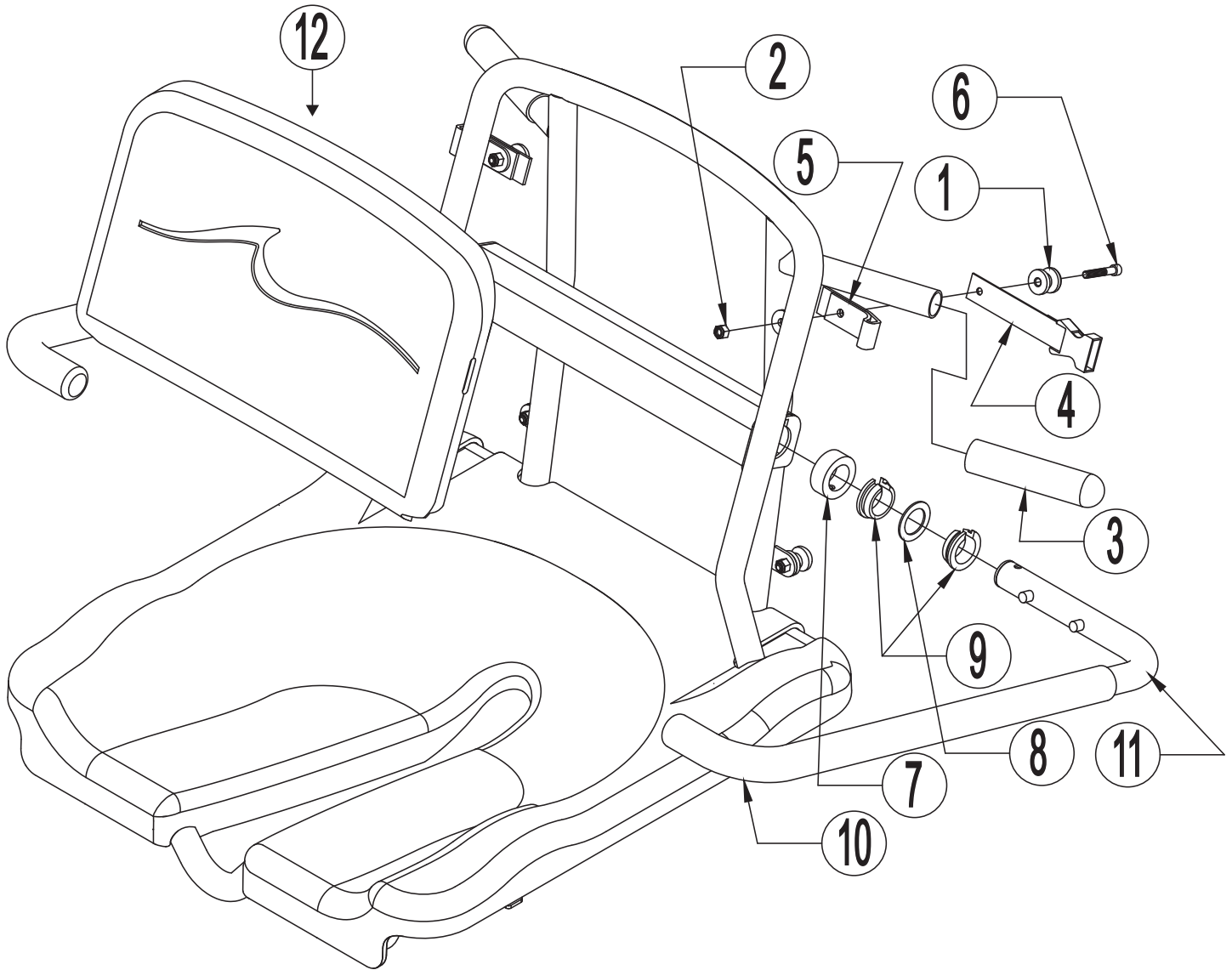


Carrier Assembly - 220079

Item	Part #	Description	Item	Part #	Description
1	220075	LH Carrier Rail Assy	12	350327	Swivel Caster LR
2	220076	RH Carrier Rail Assy	13	340512	1/4" x 1 1/4" x 4 1/2" Bumper-Black
3	220058	L/H Foot Rest Assy	14	350170	3/8" Capnut
4	220054	R/H Foot Rest Assy	15	350130	Pivot Bushing
5	220077	LR Docking Pin Retainer Assy	16	420153	Release Lever
6	350329	1/2-13 X 2 1/4 FHS HCS	17	520023	Gravity Lock Replacement Kit
7	420082	1/2" x 5/8" x 5/16" Spacer SS	18	520026	Foot Rest Replacement Kit
8	340388	L.R. Carrier Guide Wheel	19	220081	Docking Pin Cover Kit
9	350047	7/16" Std Type A Flat Washer SS			
10	350328	Locking Caster L.R.			
11	350330	1/2-13 Nylock SS			

Note:

Parts fit Model numbers 110024, 110025, 220079, and 220080 only.

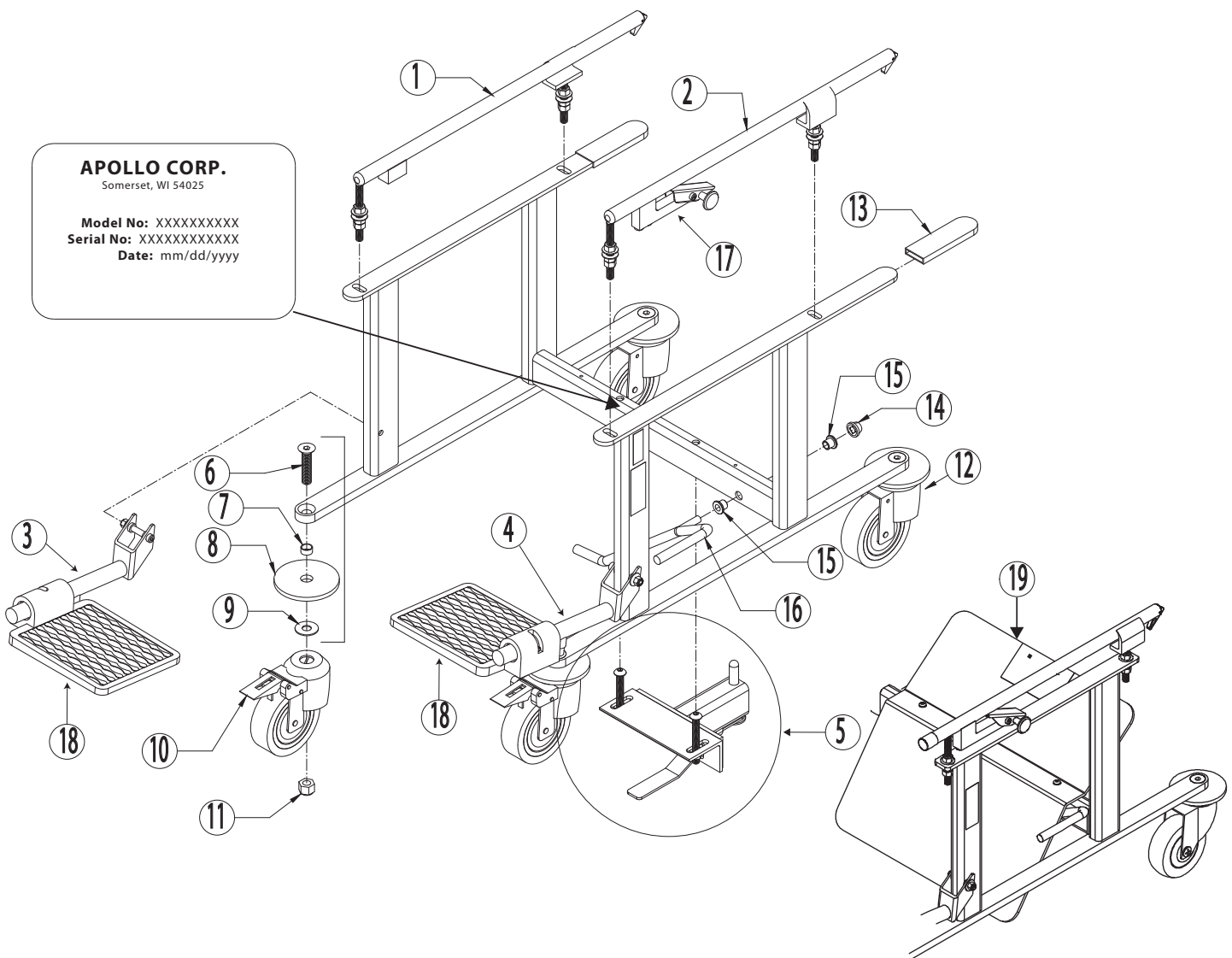


Carrier Assembly w/Scale - 220080

Item	Part #	Description	Item	Part #	Description
1	220075	LH Carrier Rail Assy	14	350170	3/8" Capnut
2	220076	RH Carrier Rail Assy	15	350130	Pivot Bushing
3	220058	L/H Foot Rest Assy	16	420153	Release Lever
4	220054	R/H Foot Rest Assy	17	520023	Gravity Lock Replacement Kit
5	220077	LR Docking Pin Retainer Assy	18	350389	5/16-18 x 1 3/4" BHSCS SS
6	350329	1/2-13 X 2 1/4 FHSCHS	19	350390	5/16-18 Acorn Nut SS
7	420082	1/2" x 5/8" x 5/16" Spacer SS	20	350490	5/8" Hole Plug Carrier
8	340388	L.R. Carrier Guide Wheel	21	350388	1/4-28 x 1/2" BHSCS SS
9	350047	7/16" Std Type A Flat Washer SS	22		L.R. Water Resistant Scale
10	350328	Locking Caster L.R.	23	220081	Docking Pin Cover Kit
11	350330	1/2-13 Nylock SS			
12	350327	Swivel Caster LR			
13	340512	1/4" x 1 1/4" x 4 1/2" Bumper-Black			

Note:

Parts fit Model numbers 110024, 110025, 220079, and 220080 only.

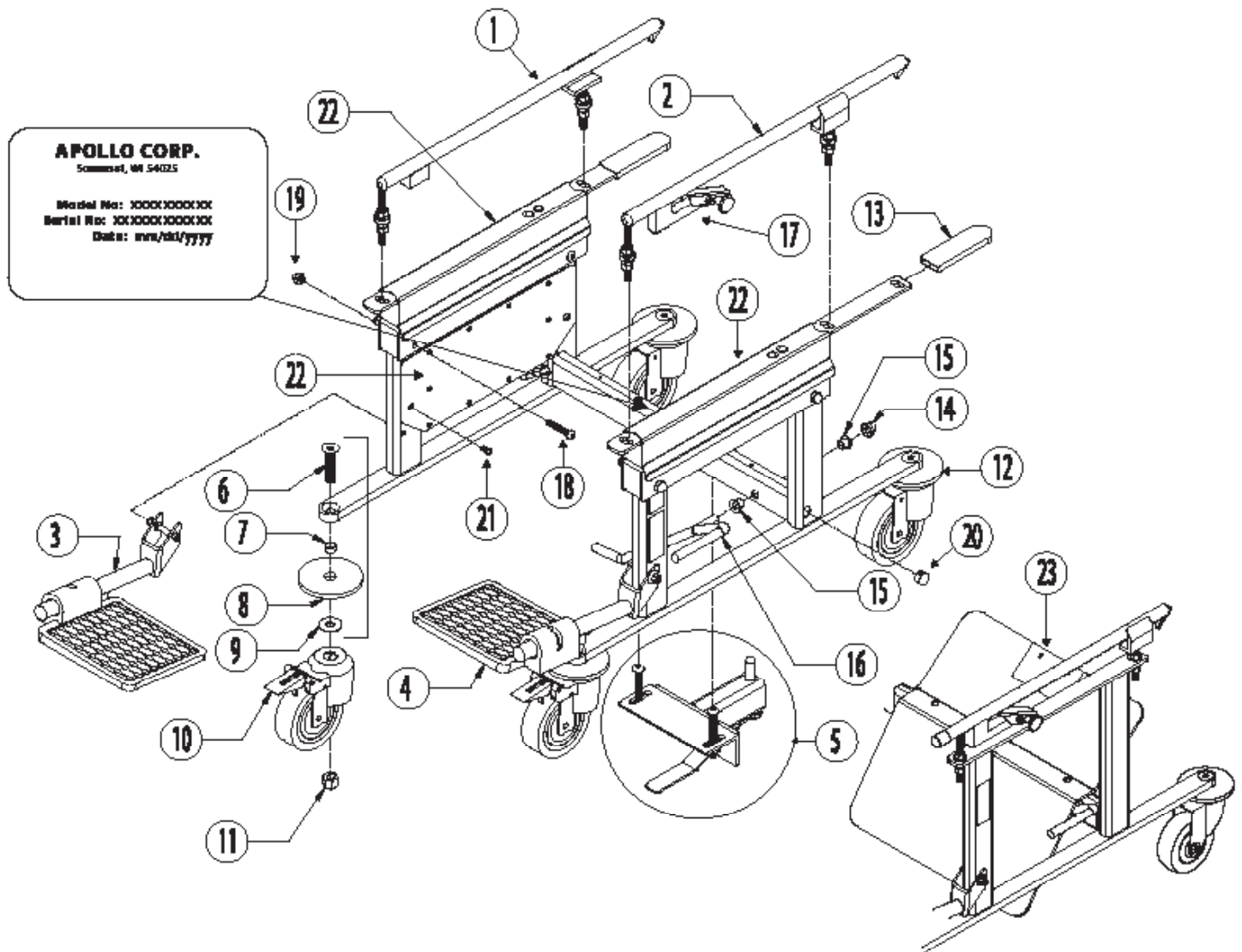

Note:

APOLLO CORP.
Somerset, WI 54025

Model No: XXXXXXXXXX
Serial No: XXXXXXXXXX
Date: mm/dd/yyyy


1 2 3 4 5 6 7 8 9 10 11 12 13 14 15 16 17 18 19

Carrier Assembly w/Scale - 220042

Item	Part #	Description	Item	Part #	Description
1	220053	LH Carrier Rail Assy	14	350170	3/8" Capnut
2	220052	RH Carrier Rail Assy	15	350130	Pivot Bushing
3	220058	L/H Foot Rest Assy	16	420153	Release Lever
4	220054	R/H Foot Rest Assy	17	520023	Gravity Lock Replacement Kit
5	220055	Pin Retainer Assy	18	350389	5/16-18 x 1 3/4" BHSCS SS
6	350329	1/2-13 X 2 1/4 FHSCHS	19	350390	5/16-18 Acorn Nut SS
7	420082	1/2" x 5/8" x 5/16" Spacer SS	20	350490	5/8" Hole Plug Carrier
8	340388	L.R. Carrier Guide Wheel	21	350388	1/4-28 x 1/2" BHSCS SS
9	350047	7/16" Std Type A Flat Washer SS	22	N/A	L.R. Water Resistant Scale
10	350328	Locking Caster L.R.	23	220073	Heel Protector Kit
11	350330	1/2-13 Nylock SS			
12	350327	Swivel Caster LR			
13	340512	1/4" x 1 1/4" x 4 1/2" Bumper-Black			

Note:


Parts do not fit Model numbers 110024, 110025, 220079, and 220080.

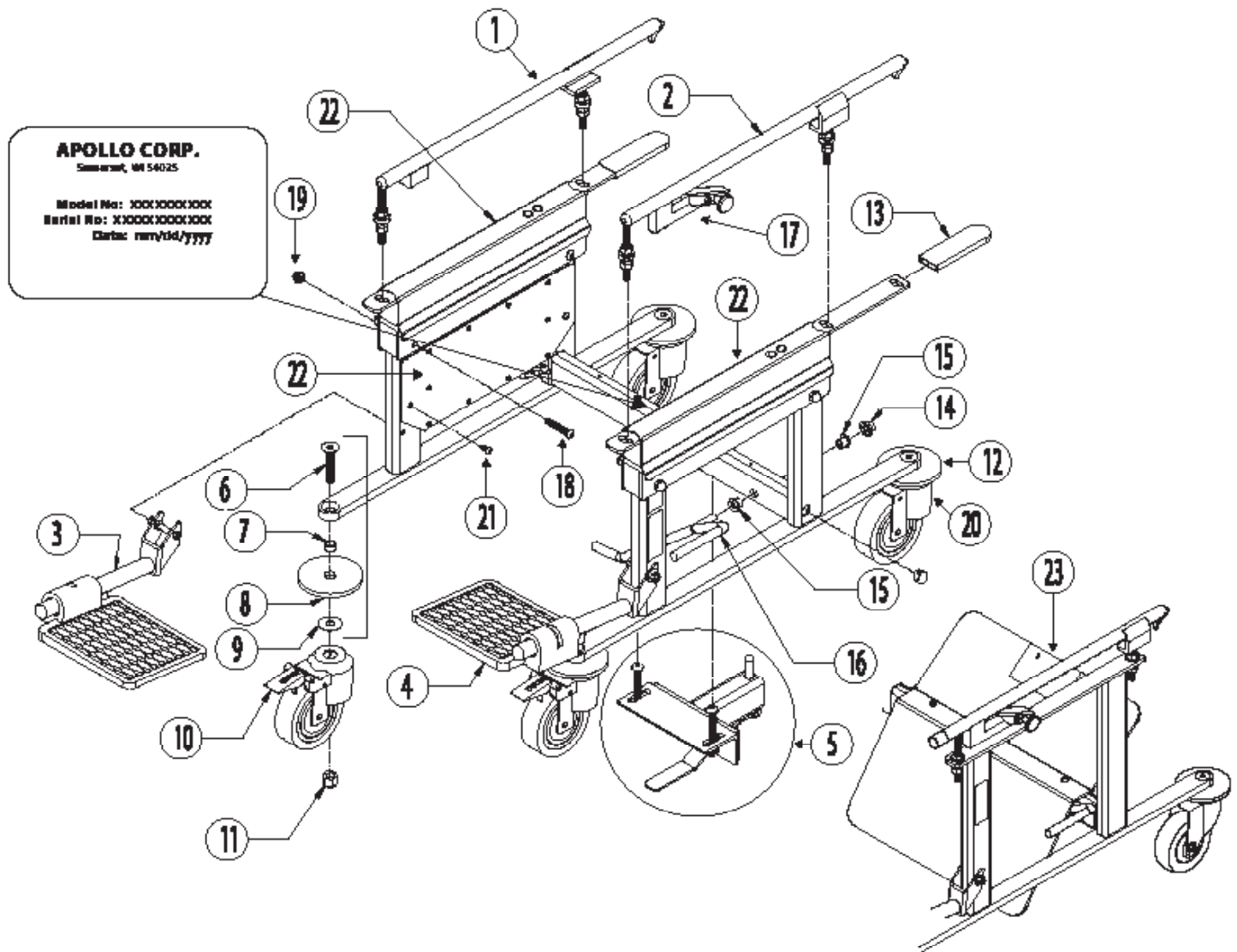

Lock-In Chair Assembly – Full Size (Tub view)


Item	Part #	Description
1	220074	Full Size Lockin Chair Assy
2	400455	Rail Chair Stop
3	350156	1/4-20 x 1" RD HD Machine Screw SS
4	400683	Standard Simplicity Tub Stop
5	350377	1/4-20 x 1 1/4" HSSS SS
6	350161	1/4" Flat Washer SS

Item	Part #	Description
7	350155	1/4" Lock Washer
8	350044	1/4-20 UNC Std Hex Nut SS

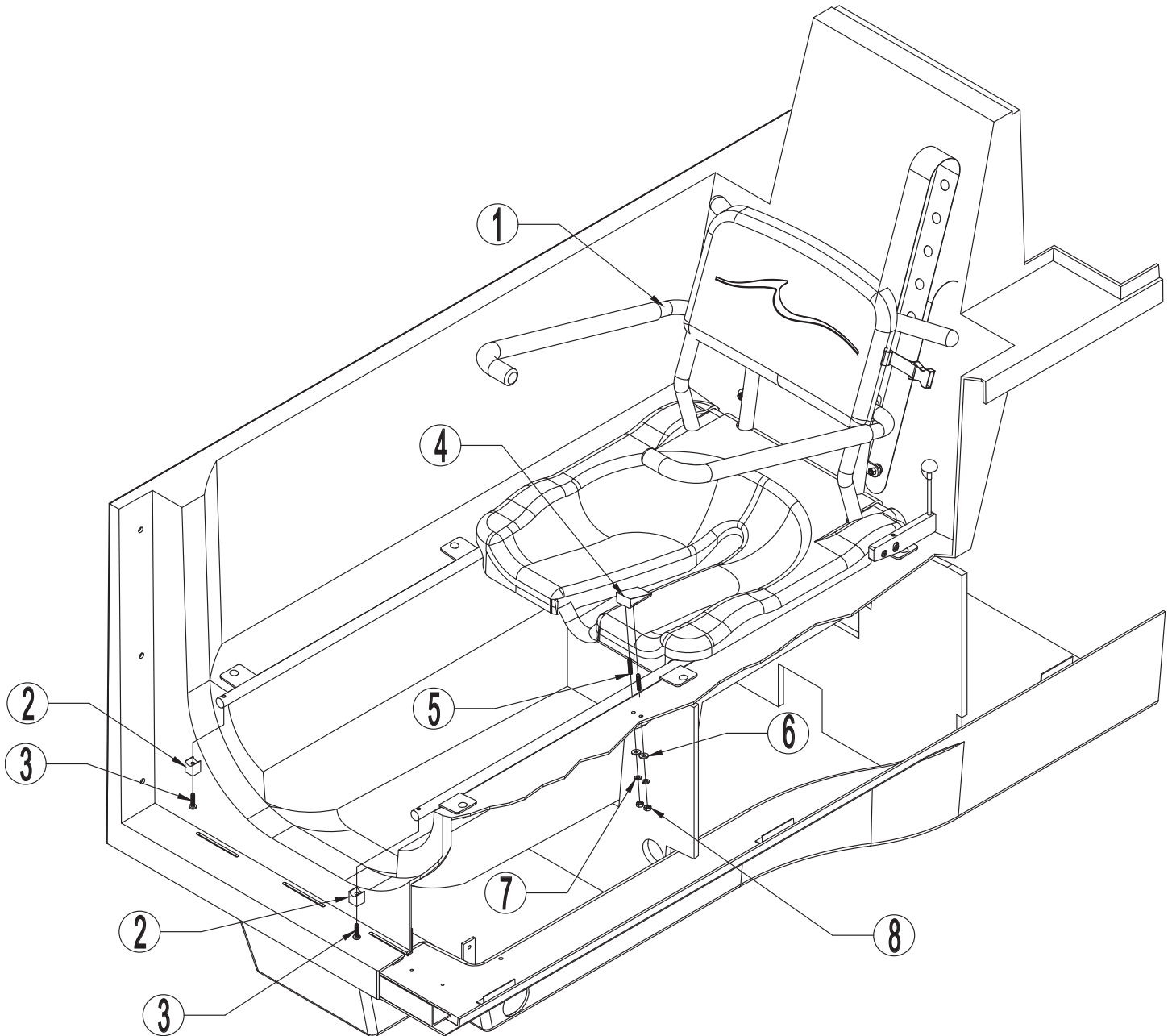

Lock-In Chair Assembly – Full Size - 220074


Item	Part #	Description	Item	Part #	Description
1	340503	SS Chair Back - Black	12	420221	Full Size Lock-In Retainer
2	350413	Knob	13	350128	10-32UNF x 3/4" Std S CRFCHMS
3	400454	Actuator Lever	14	350011	10-32 NF Nylock Nut SS
4	420202	Self Locking Lever			
5	350514	5/16" x 1" Shoulder Bolt 18-8 SS			
6	350523	1/4-20 x 1/2" Set Screw SS			
7	350121	5/16" Std SS Type A PW			
8	420201	Self Locking Lever Mount			
9	350606	.438 Dia x .005 Shim			
10	350607	.303 Dia x .005 Shim			
11	420219	Full Size Lock-In Seat Bottom			

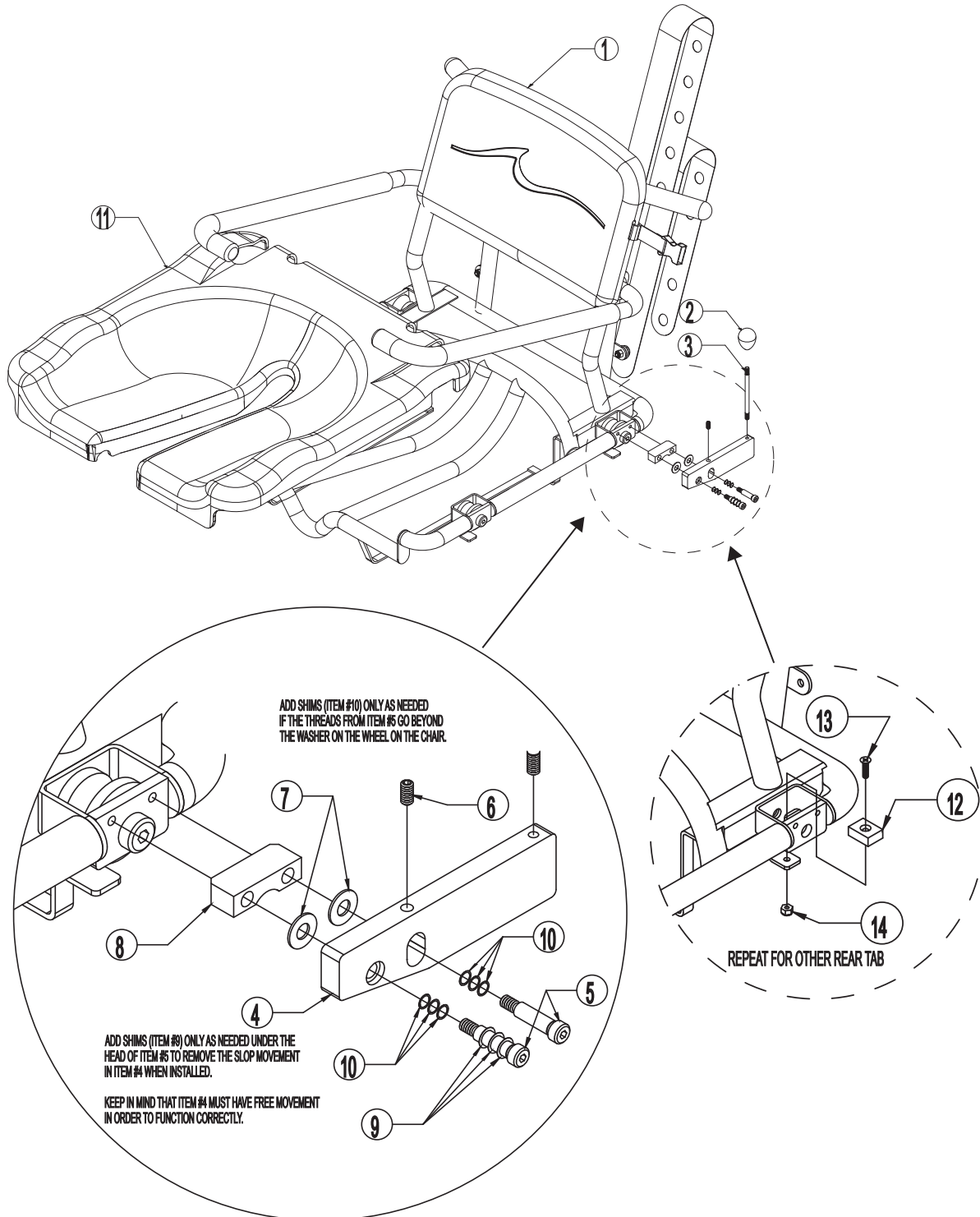

Lock-In Chair Assembly – Space Saver (Tub view)



Item	Part #	Description
1	220047	Chair Upper Assy SS Lockin
2	400455	Rail Chair Stop
3	350156	1/4-20 x 1" RD HD Machine Screw SS
4	400683	Standard Simplicity Tub Stop
5	350377	1/4-20 x 1 1/4" HSSS SS
6	350161	1/4" Flat Washer SS

Item	Part #	Description
7	350155	1/4" Lock Washer
8	350044	1/4-20 UNC Std Hex Nut SS

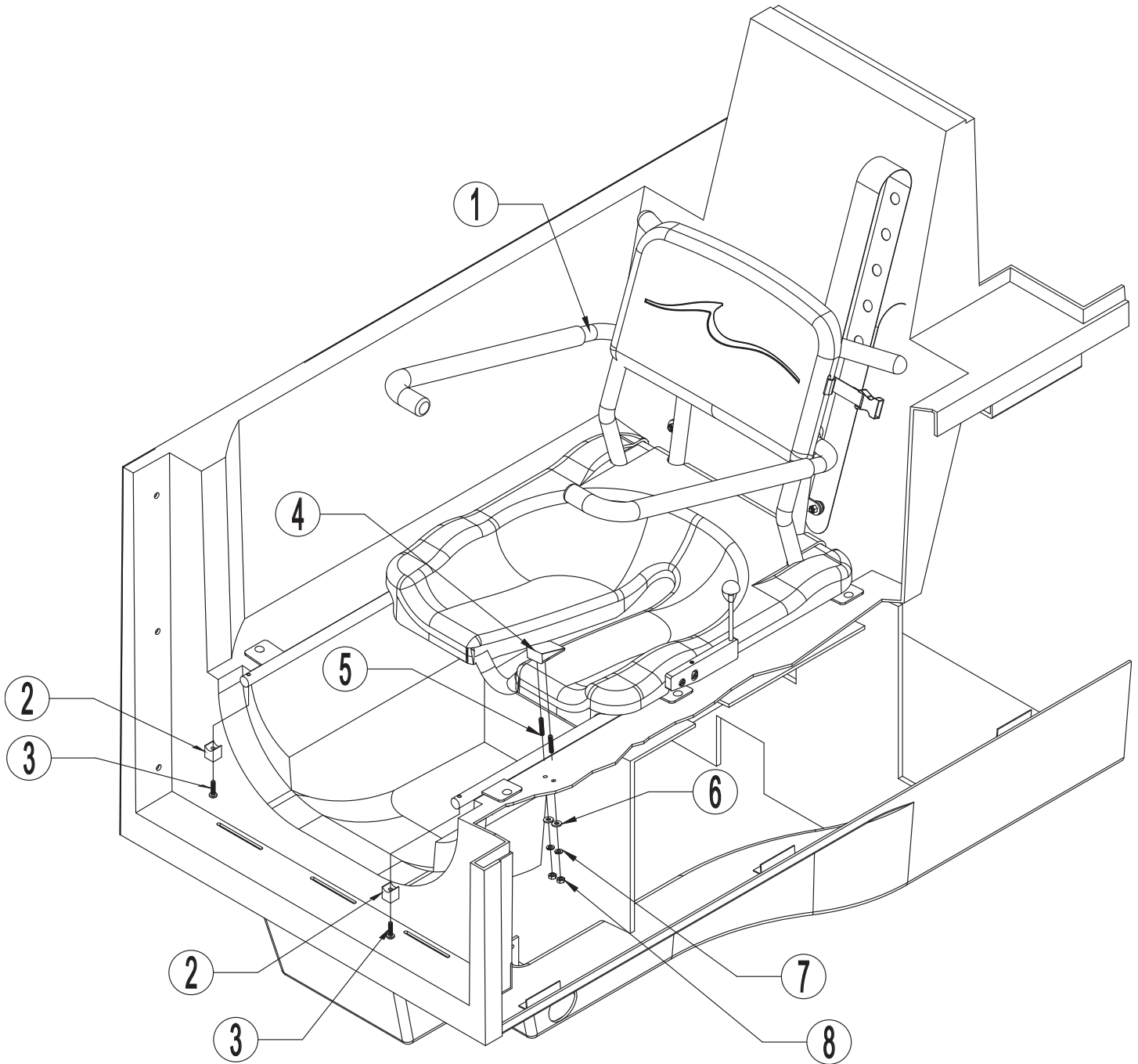

Lock-In Chair Assembly – Space Saver - 220047

Item	Part #	Description	Item	Part #	Description
1	340503	SS Chair Back - Black	7	350121	5/16" Std SS Type A PW
2	350413	Knob	8	420201	Self Locking Lever Mount
3	400454	Actuator Lever	9	350606	.438 Dia x .005 Shim
4	420202	Self Locking Lever	10	350607	.303 Dia x .005 Shim
5	350514	5/16" x 1" Shoulder Bolt 18-8 SS	11	420217	Space Save Lock-In Seat Bottom
6	350523	1/4-20 x 1/2" Set Screw SS			

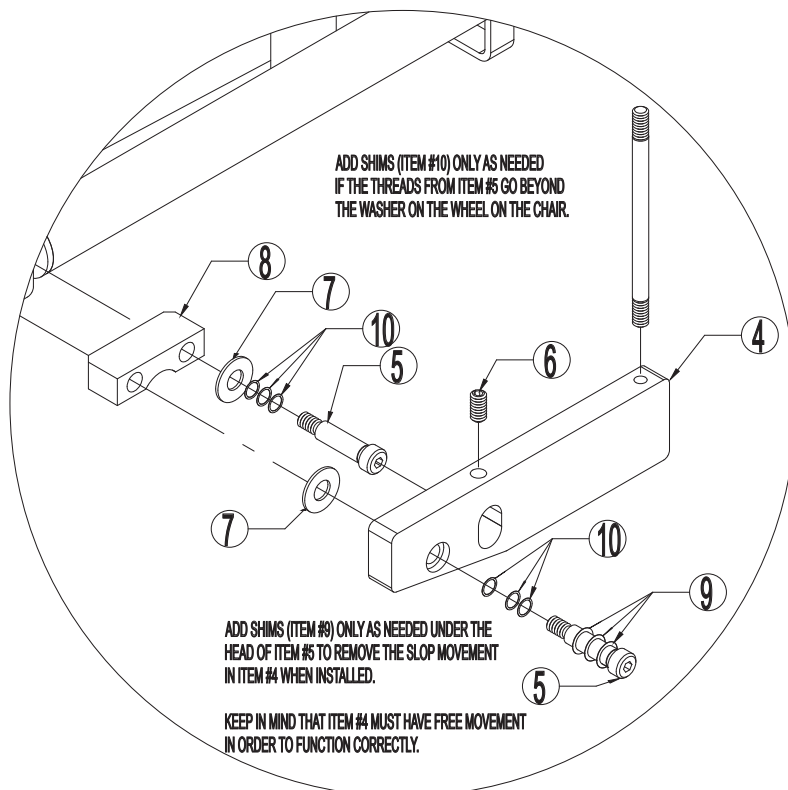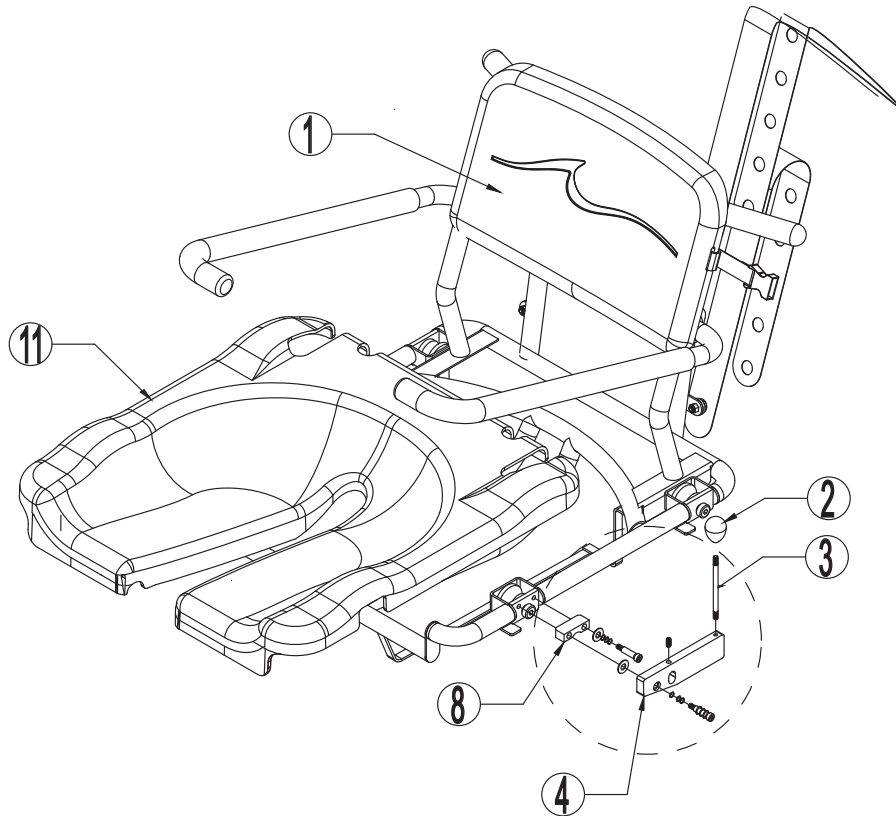


Scale Assembly (2005 - 2014)

Item	Part #	Description	Item	Part #	Description
1	330024	Scale Touch Pad with Decal	3	No Longer Available	Scale Circuit Board
2	330203	Battery Holder Size C			



Note:

Parts fit all Serial numbers except those beginning with FFI.

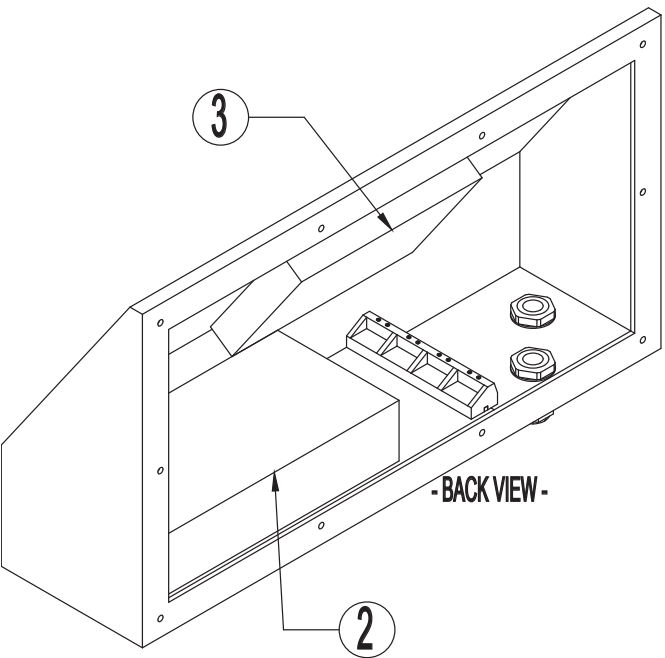

APOLLO CORP.
Somerset, WI 54025

Water Resistant Scale
Scale Serial No: XXXXXX
Date: mm/dd/yyyy

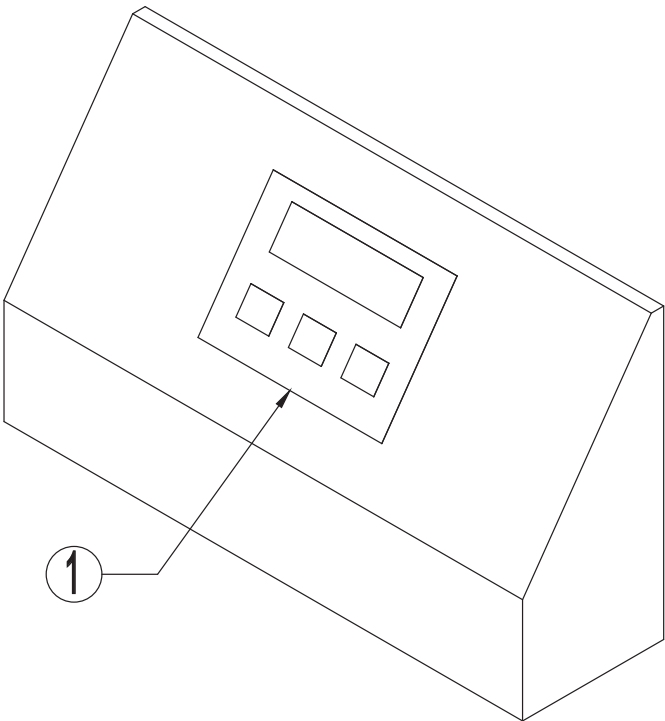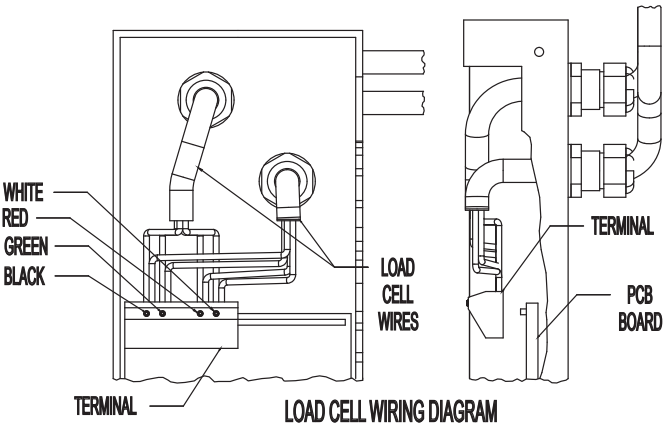

Scale Assembly - FFI (2014 - Present)

Item	Part #	Description
	220071	Scale Box with items 1, 2, and 3. Does not include back plate
1	330270	Scale Touch Pad with Decal
2	330275	Battery Holder Size C
3	330271	Scale Circuit Board

Note:

Parts only fit Serial numbers beginning with FFI.

APOLLO CORP.

Somerset, WI 54025

Water Resistant Scale

Scale Serial No: FFIXXX

Date: mm/dd/yyyy

